Garantía Limitada

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LUMEX®

- Lightweight, durable, rust-resistant frame
 - Wide seat with handles provides additional comfort
 - Available with and without backrest
 - Seat height adjusts in 1" increments for various user heights
 - 350 lb weight capacity
- EVENLY DISTRIBUTED**

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For the most updated and current information on packaging, warranties, products and specifications, please visit our website at www.grahamfield.com

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© 2014, GF Health Products, Inc.
2935 Northeast Parkway, Atlanta, Georgia 30360
Made in China www.grahamfield.com

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User Instructions and Warranty Information

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Limited Warranty

The following warranty information is a **SUMMARY** of the complete Product Warranty provided by GF Health Products, Inc. For the complete warranty, please visit our website at www.grahamfield.com.

GF Health Products, Inc. (“GF”) warrants to the original retail purchaser or lessee (either is the “Customer”) only, that it will replace or repair components, at GF’s sole discretion, that are defective in material or workmanship under normal use and service. The warranty is for parts only and does not include labor or costs of shipping. All warranties are conditioned upon the proper use of the product strictly in accordance with applicable GF instructions. Product provided to users as rentals must be serviced by the provider and inspected prior to delivery. Each user must be properly trained on the operation and safety of this product prior to use. Within the guidelines set forth in this document, the frame is warranted for five (5) years and the normal wear components such as tips, wheels/casters, and handgrips are warranted for three (3) months. The applicable warranty period shall commence from date of shipment to the Customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of the warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Dealer from whom the Customer purchased the product within the applicable warranty period. If there is not a Dealer, you must contact GF directly by calling (678) 291-3207, sending a fax request to 770-368-2386 or by e-mailing a request to cs@grahamfield.com. Specific directions will be provided by the Customer Service Representative.

EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following: 1) Defects, damage, or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; 2) Products not installed, used, or properly cleaned and maintained; and 3) Labor or shipping charges. Other exclusions apply.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER
THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED.

For the complete warranty for this product, please visit our website at www.grahamfield.com.

Instructions for Use – Please Read All Instructions Before Use.

Setup

Installing the Back

(Only for 7921A-4)

1. Place bath seat on its side on a flat working surface.
2. Locate the channel guides (4 each) on underside of seat. Align backrest support tubes with channel guides.
3. Slide backrest seat support tubes through channel guides until seat support tube push buttons protrude fully through channel guide holes.

NOTE: Ensure seat support tube buttons protrude fully through channel guide holes.

Caution: Backrest is not designed to support the full weight of the user.

Adjusting Leg Height

Legs are already installed on the bath seat.

To adjust leg height:

1. Press push button and slide leg extension up or down to desired adjustment hole.
2. Ensure push button fully protrudes through the leg adjustment holes of the leg extension.
3. Repeat STEPS 1 and 2 for the other three legs.

NOTE: An audible “click” will be heard when push button securely protrudes through the adjustment hole.

NOTE: Ensure all four legs are adjusted to the same height and that all height adjustment buttons protrude fully through the adjustment holes before use.

Maintenance

- Use a mild soap and water solution, rinse with clear water, then dry with a soft cloth. Ensure that product is well rinsed before using.
- Never use organic solvents, abrasive cleaners, or scouring pads on any part of the product.
- Check fit and tightness of all nuts, bolts and knobs weekly to ensure that product is safe to use.
- To guarantee the best service from your product, always specify Lumex replacement parts, available from your GF Health Products, Inc. authorized distributor.

For more information regarding GF Health Products, Inc. products, parts and services, please contact your distributor or visit us online at www.grahamfield.com.

Safety Guidelines – Please Read Before Use

NOTE: Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

WARNING

WARNING: DO NOT install or use this equipment without first reading and understanding these instructions. If you are unable to understand the Warnings, Cautions or Instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

WARNING: Item has a maximum weight capacity of 350 lb. **EVENLY DISTRIBUTED.**

WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.

WARNING: ALWAYS inspect the bath seat to ensure that it is properly positioned and stable before using. DO NOT use if bath seat is wobbly or unstable.

WARNING: If components are damaged or missing, contact your GF Health Products, Inc. distributor immediately. DO NOT use substitute parts.

WARNING: Check leg tips for rips, tears, cracks or wear. If any of these conditions exist, replace leg tips IMMEDIATELY.

WARNING: DO NOT use this as a transfer bench or climbing device.

WARNING: To avoid falling or tipping: Have someone help you until you know the balance points of your Bath Seat and how to avoid tipping. Do not put downward weight on one side or corner of the Bath Seat. Do not put downward weight on the backrest. Do not reach or lean if you must shift your weight to do so. Ask for help or use a device to extend your reach. Never try to reach an object if you must move forward in your seat and your buttocks do not stay in full contact with the seat. Never lean backward to reach an object.

WARNING: All four leg tips MUST be in contact with shower/tub floor at all times.

WARNING: Ensure that the four chair legs are adjusted to the same height and that the push buttons fully protrude through their respective adjustment holes BEFORE use.

WARNING: Only use GF Health Products, Inc. accessories with this product.

WARNING: This product should only be used with tub floors wider than 18 ¾ inches.

WARNING: Users with limited physical capabilities must be supervised or assisted when using the bath seat.

WARNING: Make certain ALL screws and nuts are tight at all times.

WARNING: Notice for California Customers
California Proposition 65 WARNING: This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.

WARNING: DO NOT submerge product in water, otherwise damage may result.

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ADVERTENCIA: Utilice solamente accesorios de GF Health Products, Inc. con este producto.