

## 7150/7155 BARIATRIC RECLINER

### **Warning, Watch for Pinch Points**

Keep fingers, arms, legs and objects clear when any mechanism is in motion. Make sure all persons and objects are clear when raising or lowering the back.

The Bariatric Recliner Chair is equipped with 2 brake casters (back of chair) and 2 steer casters (front of chair). When entering or exiting the chair make sure the brakes are engaged. The steer caster will lock for pushing the chair in a straight line.

To operate the recliner pull up on the release handle and push down on the back to the desired position then release the handle. As the back comes down the foot rest will go up.

**Caution:** Before using any cleaning product, test it on the underside of chair. Never use any volatile or flammable fluid (thinner, gasoline, etc.) that could cause discoloration or deterioration.

### **Cleaning**

Use mild soap and water or a nonabrasive hospital-grade detergent disinfectant to clean the chair. Wet a cloth with the cleaning product and wring dry. Then gently wipe off the chair.

### **Periodic Inspection and Maintenance**

The amount of maintenance required on the chair will be dictated by its use.

Depending on usage, inspect the chair after one to two weeks of usage and then at a minimum of every six months.

Before two weeks and then every six months minimum thereafter

Check all the bolts to be sure that none have worked loose.

Check that pins are securely fastened.

Check all welds.

Gendron, Inc  
520 W. Mulberry Street – Suite 100  
Bryan, Oh 43506  
Phone (800) 537-2521



Model No.....

Serial No.....

Date of purchase.....

BUILT, INSPECTED & PACKED BY: ..... DATE BUILT: .....

## LIMITED WARRANTY

This limited warranty is extended to the original purchaser/user and cannot be transferred.

The Bariatric Recliner welded chair frame is warranted for 5 years against defects in material and workmanship. Upholstered components, seat, back, leg support, arms are warranted for 1 year against defects in material and workmanship.

Defective parts will be repaired or replaced by the company or its appointed agent. The warranty does not cover damage caused by misuse or negligence, nor does it cover defects or damages by use of unauthorized parts or service by an unauthorized person.

This warranty gives you specific legal rights, and you may also have other rights that vary from state to state.

Do not return merchandise without prior authorization. All correspondence relating to this warranty must indicate Model number and Serial number and be shipped to:

**GENDRON, INC.**  
**520 W Mulberry St – Suite 100**  
**Bryan, Ohio 43506**

## NOTICE

**FREIGHT DAMAGE** -The carrier who delivers merchandise to your door is responsible for loss and damages. Acceptance of the shipment by you is acknowledgment that all articles delivered were in good condition and properly packed. Therefore, all claims for loss or damage must be filed immediately with the freight carrier. Then notify us. We will mark our records accordingly. After settling claim, damaged merchandise will be picked up by the freight carrier and returned. Should you need assistance with the claim, call our customer service at 800-537-2521.

**RETURN GOODS** - Within 60 days from date of purchase returns will be subject to a handling and re-stocking charge plus any additional costs resulting from damage or use.

After 60 days from date of purchase: Gendron Inc. reserves the right to not accept return goods for any reason. If accepted, the amount of credit will be at the discretion of Gendron Inc. Returned merchandise without prior authorization will not be accepted.