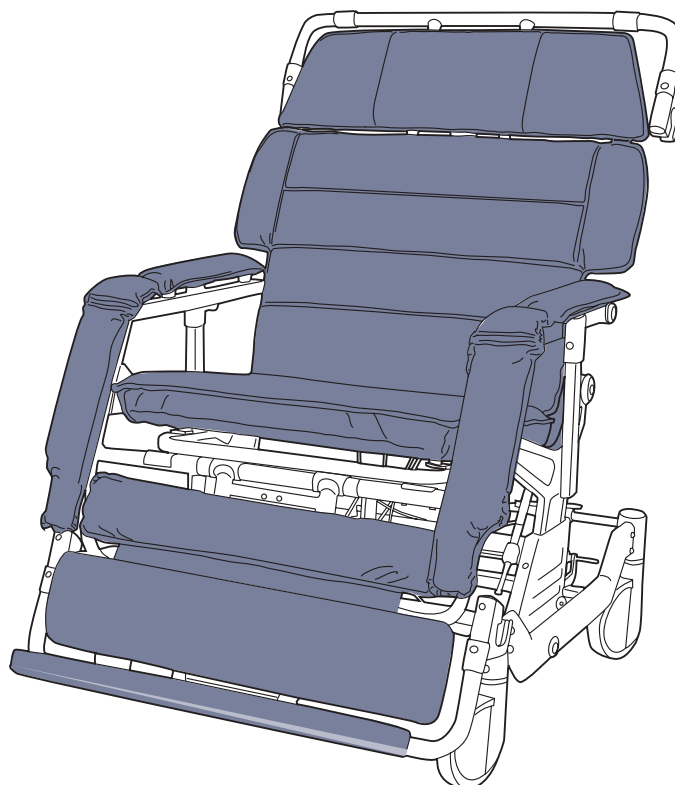
**HUMAN CARE**  
Your Life. Your Way.

**H&R**

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## Contents

---

<b>User manual</b>	<b>4</b>
Safety Instructions	4
Symbols	4
Intended Use	4
General Care and Use	4
General Warnings	5
Warnings	5
Recommendations	5
Product Benefits	5
Product Description	5
Prior to Usage	6
Components	6
Safety Warnings	6
<b>Usage Guidelines</b>	<b>8</b>
Adjusting the Seat Height	8
Levers	8
Adjusting the Headrest	8
Adjusting the Head Support	9
Adjusting the Armrests	9
Adjusting the Footrest	10
Caster Functions	10
Adjusting the Brackets	10
<b>Operating the Chair</b>	<b>11</b>
Tilting	11
Sitting to Supine	11
Supine to Sitting	11
Trendelenburg	12
Cleaning Recommendations	12
Material	13
Troubleshooting	13

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## Safety Instructions

### Important!

You must read the user manual for your device prior to usage. Keep this booklet and information handy for future use.

## Symbols

**WARNING** symbol indicates a potentially hazardous situation which, if not avoided, could result in serious injury to the user or damage to property and/or the device itself.

**INFORMATION** symbol indicates recommendations and information for proper, trouble-free usage.

**WEIGHT CAPACITY** symbol indicates maximum user weight for the product. Do not exceed this weight in any circumstance.

**CHLORINE** symbol indicates usage of approved chlorine on product is permitted.

Symbol indicates hand washing only.

Symbol indicates item may be wiped with approved disinfectant.

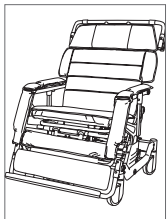
**CE MARKING** symbol indicates product meets requirements of EC directives and can be sold in North America.

**RECYCLE** packaging according to local regulations.

## Intended Use

### Convertible Chair

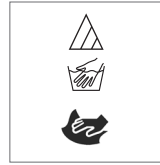
Human Care's Convertible Chair is a mechanical positioning chair that can be adjusted to various positions. The device is used to alter postural positions and to provide a means of transferring a patient horizontally from a height adjustable bed or trolley to the chair and vice versa. The chair can then be transformed from a supine position into the sitting position without ever lifting the patient.

Its construction is made of steel tubular components, on to which is fitted foam-filled upholstery. The mechanism allows the chair to transform from the horizontal supine position into a chair, which can then be tilted. The reclining function allows the chair and patient profile to change from a sitting position to a horizontal or supine position suitable for transferring. The Tilt-in-Space function allows the I-400 / I-700 to be independently inclined to the rear (Tilt-in-Space).


## General Care and Use


Check the Convertible Chair prior to each use to ensure it is not damaged in any way. Do not use if there are any signs of wear and damage on part of the chair.

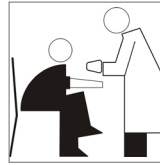Follow cleaning instructions as indicated.

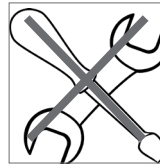Only qualified persons should perform patient transfers. Use caution during transfer.

Do not attempt to repair the chair yourself or modify it in any way.

Do not use this equipment to transfer patients in a moving vehicle of any type.

Use only the Human Care patient transfer system designed for use with the I-series positioning and transfer chairs.

This product is intended for indoor ambient environments. It is not intended for use in high moisture areas (i.e. showers, pool therapy), extreme temperature, humidity or pressure environments.

Facility administrator and/or risk management department should approve all patient handling protocols associated with the Convertible chair prior to use.

Some patients' physical limitations or weights present the potential for injury. When in doubt that an attendant or patient may be injured, ensure additional attendants are present.

Prior to using on a patient, it is strongly recommended that training or practice with other caregivers is done until the user is comfortable with using the product.


Assess the patient's ability to handle the move prior to using the Convertible Chair.

## General Warnings

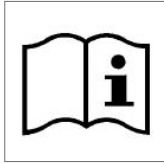The decision to use the Convertible positioning and transfer chair must be made by a trained professional. In all cases, this chair can only be articulate when the patient is attended.

If you are unable to understand the warnings, cautions and/or instructions, contact a healthcare professional, dealer or technical personnel (if applicable) before attempting to use this equipment. Using this equipment without fully understanding it may result in injury or damage.

Therapists and other healthcare professionals should determine if the patient requires a seat positioning strap prior to use to ensure safe operation of this equipment by the user. Serious injury can occur in the event of a fall.

## Warnings

**DO NOT USE** the product if,

**Weight capacity is exceeded.**

**Product is damaged in any way.**

**You have not read and understood all transfer procedures.**

**You have not read and understood all warnings and instructions.**

## Recommendations

**Do not use Convertible Chair without adequate prior training.**

**Increase safety by evaluating transfer environment prior to transfer.**

**There is always risk when transferring a patient. Evaluate patient ability prior to transfer.**

**To avoid risk of caregiver injury, have two caregivers perform the transfer.**

## Product Benefits

### For the caregiver/facility

- Simplifies job conditions - efficient and easy to use
- Reduces patient length of stay due to early mobilization
- Injury prevention - driving force for SPH initiatives

### For the patient

- Safe and effective transport
- Increased comfort as no sling application is required
- Faster recovery

## Product Description

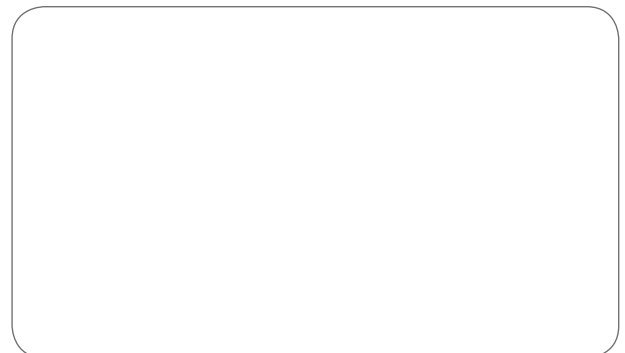
### Technical Specifications

Item	Part no.	Seat Width	Overall Width (cm/inch)	Weight Capacity
I-400	B1041-TS-GEL	21"/53cm	30"/76cm	181kg/400lbs
I-700	B1071-TS-GEL	25.5"/65cm	35"/89cm	318kg/700lbs

Item	Part no.	Overall Length (Prone Position)	Overall Length (Wheelchair Position)
I-400	B1041-TS-GEL	72.5" - 78.5"/184-199cm	45.5" - 62.5"/115-159cm
I-700	B1071-TS-GEL	72.5" - 78.5"/184-199cm	45.5" - 62.5"/115-159cm

Item	Part no.	Overall Height (Prone Position)	Overall Height (Wheelchair Position)
I-400	B1041-TS-GEL	27.5"/70cm	51" - 56"/130-152cm
I-700	B1071-TS-GEL	27.5"/70cm	51" - 56"/130-152cm

## Authorized Dealer

## Prior to Usage

Before use verify that:

All casters work properly (4-caster lock, 4-caster free, steering lock).

The equipment reclines smoothly and stops in any desired position.

The height and angle of the footrest and the height, angle and depth of the headrest can be adjusted properly.

The side rails can be locked and released reliably.

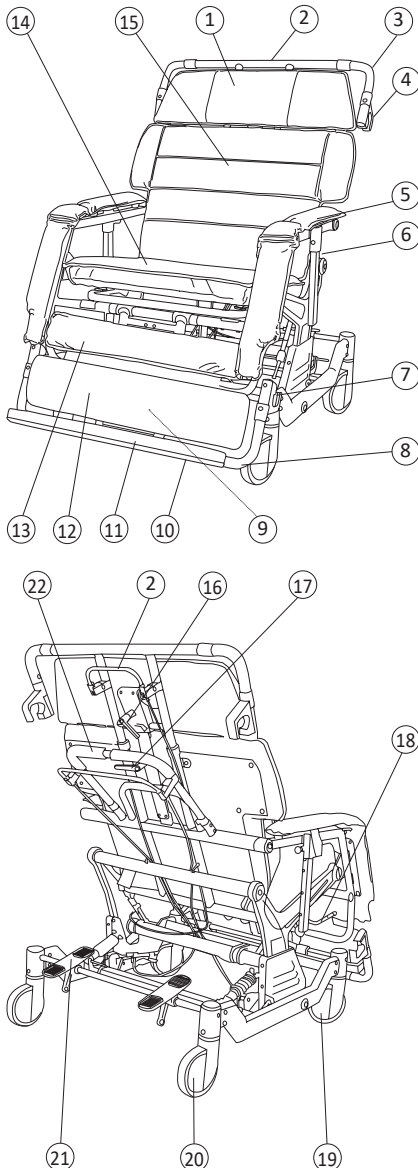
The equipment tilts smoothly and stops in any desired position.

All height adjustment screws are securely tightened.

## Components


1. Headrest
2. Headrest Handle
3. Chair Bracket (Headrest)
4. Plastic Bearing
5. Armrest
6. Armrest Height Adjustment Knob
7. Plastic Bearing
8. Chair Bracket (Footrest)
9. Footrest Height Adjusting Lever (Back Side)
10. Footrest Guard
11. Footrest
12. Leg Rest
13. Calf Pad
14. Seat
15. Backrest
16. Headrest Depth/ Angle Adjusting Lever
17. Headrest Height Adjusting Lever
18. Leg Rest Angle Adjusting Lever
19. Front Steering Caster
20. Rear Caster
21. Caster Control Pedal
22. Push Handle


## Safety Warnings

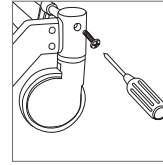
You must read and understand the warnings and cautions in this manual to use the Convertible I-400/I-700 positioning and transfer chair safely. Failure to observe them can lead to an accident and can seriously injure the patient or caregiver.

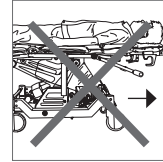
After any adjustment, repair or service and before use, ensure that all attaching hardware is tightened securely.

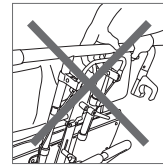
Do not attempt to move in fully reclined or nearly reclined position.

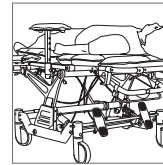
Do not push or pull the chair by the headrest handle.

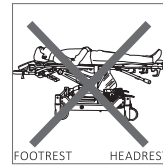To maintain maximum stability when reclining/ inclining, position the patient with arms and legs inside the chair and clear of moving parts.

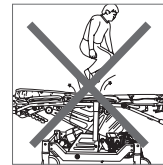Do not confuse the headrest and footrest when placing the patient in the prone position.


Do not jump up and down on the back or seat cushion.


Do not use stairs or escalators to move an occupied chair in any position.

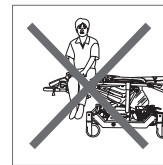
Ensure that all persons and objects are clear of the chair when you recline/incline the equipment.

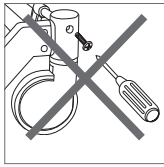Never sit or lean on the headrest or leg rest when the chair is in the supine position.

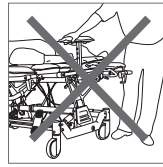Never disassemble or remodel the Convertible chair. Use only parts and/or accessories authorized by the manufacturer for repair.


Do not attempt to move the Convertible Chair by holding the armrest.


The Convertible chair is not designed to properly support a patient receiving CPR. This will change the center of gravity and may cause the equipment to tip over.

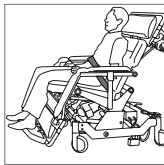
The Convertible Chair is designed for a single occupant. Never allow two or more people to rest on the chair, nor allow any heavy objects to be placed on the chair.

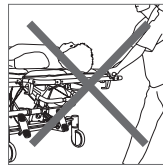
It must be determined prior to use by a licensed therapist or health care professional if a seat positioning strap must be used with the chair.

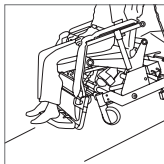
The Convertible Chair is not designed to be used as a stretcher, weight training or rehabilitation apparatus. Never use it for such purposes.

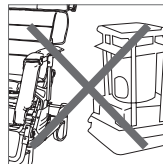
Take special care when going up and down a ramp. Never park the chair on a slope. Only operate chair on handicap grade access ramp.

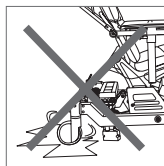
Do not use near an open flame (stove, cigarette, etc.) or an extreme heat source.

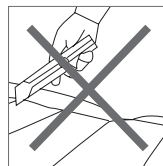
Do not use on gravel, frozen roads or any surface that is extremely slick or wet.

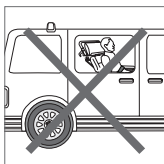
Never apply cutters or sharp edged objects to any of the cushions.

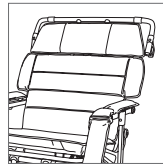
This equipment has no tie down system approved for transporting an occupied chair in a moving vehicle of any type. Do not use for transport in vehicles of any kind while the equipment is occupied.

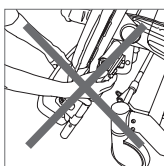
Do not use the Convertible Chair with any missing or abnormal parts.

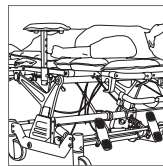
Do not attempt to lift the chair by holding the leg rest assembly.

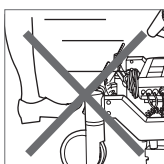
Always raise the armrests when the patient is in the supine position. The armrests act as side rails.

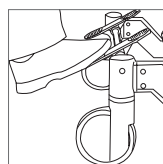Do not attempt to use the castor locks to stop the Convertible Chair in motion. These locks are not designed to be braking devices.

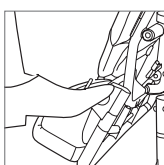Take special care to lock the casters securely.

Do not use the footrest as a step when you sit on or get off the Convertible Chair.

Take note of the maximum user weight. People who weigh more than the maximum user weight cannot use these models.

**I-400 MAX 400lbs/181kg**

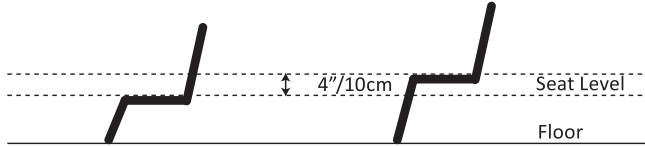
**I-700 MAX 700lbs/317.5kg**

# Usage Guidelines


## Adjusting the Convertible Chair Seat Height

**Seat Height Adjustment**


 **The Convertible Chair is adjustable within a 4"/10cm range. Do not set height above 4"/10cm mark.**

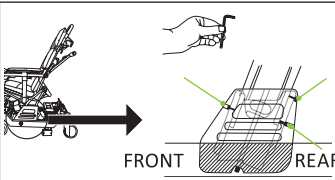
1. With a 5mm Allen Wrench loosen the six bolts on the height adjustment diecasts on both the left and right side. Be careful not to unscrew the bolts.

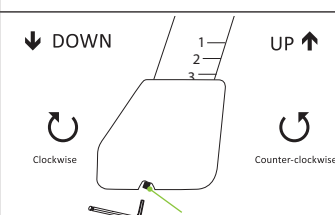2. Using the Allen Wrench, rotate the bolt counter-clockwise to raise and clockwise to lower alternating between left and right sides in 1/2"/1.3cm increments. Repeat until seat height reaches desired height.

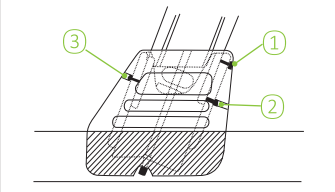3. Tighten bolts ① and ② on both sides. After tightening these bolts, tighten bolt ③ on both sides.

 **Make sure all bolts on both sides are fully tightened after finishing height adjustment of seat.**

## Convertible Chair Levers

**The Convertible Chair has three adjusting levers:**

1. Headrest height adjusting lever
2. Headrest depth/angle adjusting lever
3. Footrest height adjusting lever.

**These levers which are in the same form have two functions:**


**Function 1: Tighten / Loosen;**

**Function 2: Change lever position in a tightened state**

 **Never change the lever position in a loosened state - the lever cannot be adjusted unless it is tightened.**


**Lever Function 1**

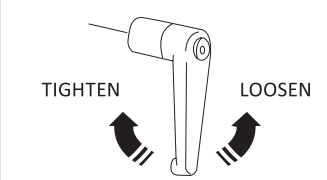To tighten lever for adjustment turn clockwise. When tightened, make adjustment according to the instructions for applicable lever. To loosen lever turn counter-clockwise.

**Lever Function 2**

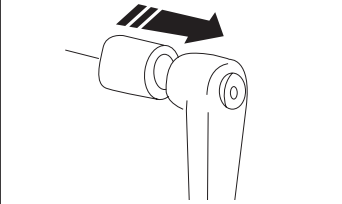Adjust the lever position by pulling lever out while in a tightened state.


## Adjusting the Convertible Chair Headrest


**Depth/Angle Adjustment**

 **When loosening the headrest depth/angle adjustment lever, be sure to hold the headrest securely in place. Loosening the lever with the headrest unsupported may cause the headrest to drop abruptly and an injury or accident may occur.**

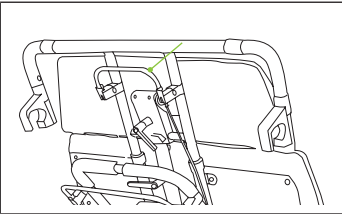
1. Hold the headrest handle securely with one hand.

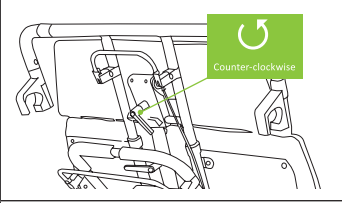
2. Loosen the headrest depth/angle adjusting lever with the other hand by turning it counter-clockwise.

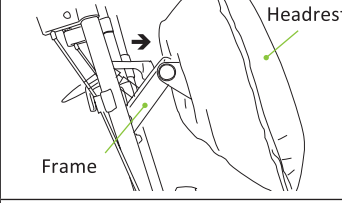
3. Position by pushing out the headrest (both depth and angle) when the adjusting lever is in its loose position.

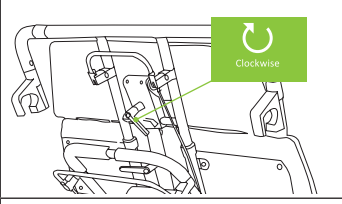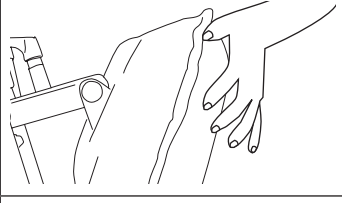
4. Once the depth and angle are correctly positioned, secure the headrest in place by turning the lever clockwise.

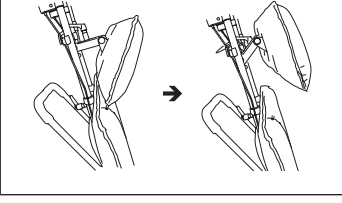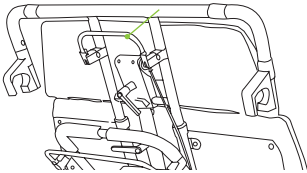
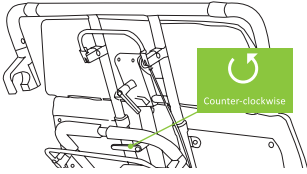
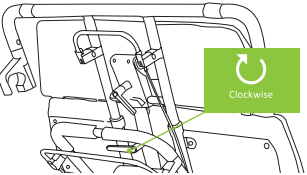
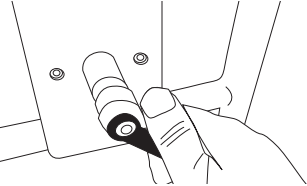
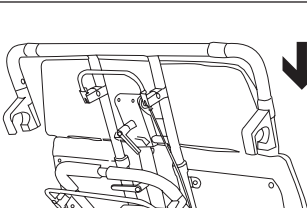
5. To return the headrest to its original angle, hold the headrest securely in one hand so that it does not fall abruptly and loosen the lever by turning it counter-clockwise.

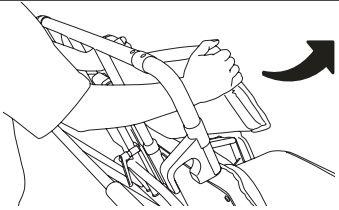
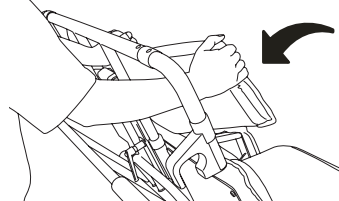
6. Move the headrest from the existing position to the original position and secure in place by turning the adjustment lever clockwise.

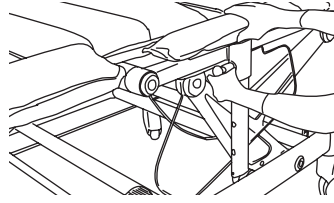
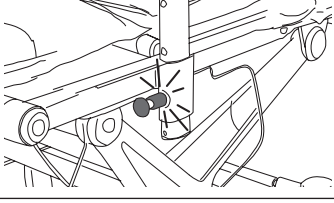
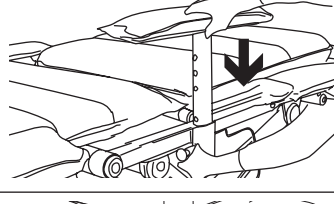
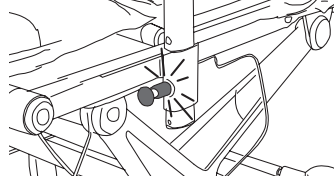
 **When moving the headrest back to the home position, ensure that nothing comes between the headrest and the chair frame.**

Raising/Lowering the Headrest	
<p> <b>Always use both hands when raising/lowering headrest otherwise an injury or accident may occur.</b></p>	
<p>1. To raise the headrest, first hold the headrest handle with one hand.</p>	
<p>2. Loosen the headrest height adjusting lever with the other hand by turning it counter-clockwise</p>	
<p>3. Raise the headrest slowly. When the headrest reaches the desired height, tighten the lever fully clockwise.</p> <p><b>Note: At the correct height, the patient's neck is visible.</b></p>	
<p>4. To return the headrest height to its original position, hold the headrest handle with one hand and loosen the height adjustment lever with the other hand.</p>	
<p>5. Once loosened, lower the headrest slowly to its lowest position and tighten the lever fully.</p>	
<p> <b>IMPORTANT! Be sure to restore the headrest to the lowest position before transforming the Convertible Chair to the supine position otherwise the patient's head could get caught between the back and the headrest resulting in injury or accident.</b></p>	

## Adjusting the Convertible Chair Head Support



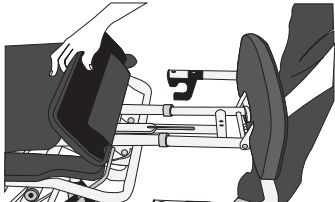
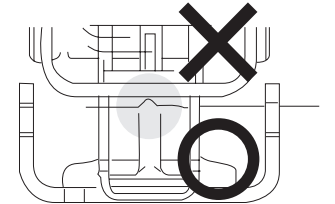
<p> <b>When moving the headrest back to the home position, ensure that nothing comes between the headrest and the chair frame.</b></p>	
<p>1. Hold the middle part of the headrest with one hand and turn one side at a time slowly inward with the other hand.</p>	
<p>2. To return the headrest to its original position, hold the middle part with one hand and pull one side at a time slowly towards you with the other.</p>	

## Adjusting the Convertible Chair Armrests

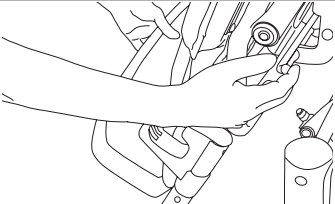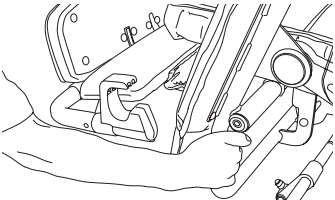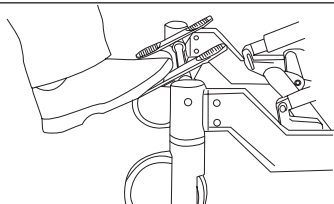
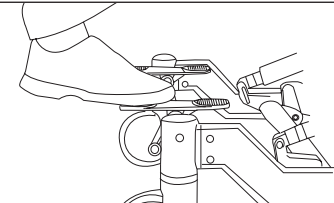
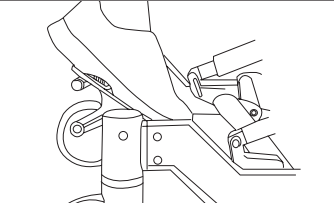
<p>The armrests can be used as siderails by raising them.</p>	
<p> <b>Warning: Ensure that armrests are securely locked in place. An unsecured armrest may fall abruptly and an injury or accident may occur.</b></p>	
<p>1. Pull the armrest height adjustment knob with one hand while simultaneously raising the armrest with the other hand.</p>	
<p>2. Once the side armrest is at the desired height, release the height adjusting knob and the armrest will lock into place.</p>	
<p>3. To lower the armrest, pull the armrest height adjustment knob with one hand while lowering the armrest with the other.</p>	
<p>4. Once the side armrest is at the desired height, release the height adjusting knob and the armrest will lock into place.</p>	

# Usage Guidelines

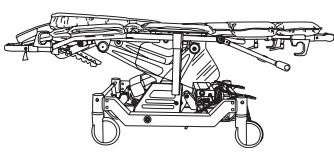
## Adjusting the Convertible Chair Footrest






<p><b>Adjusting the Height</b></p> <p> <b>Use both hands when raising or lowering the footrest otherwise an injury or accident may occur.</b></p>	
<p>1. Hold the footrest guard handle with one hand and loosen the footrest height adjusting lever with the other.</p>	
<p>2. Raise or lower footrest slowly. When it reaches the desired height, tighten the lever fully. Caution: Position the footrest as shown - with the lever up, the leg rest may not be adjusted properly.</p>	
<p><b>Adjusting the Angle</b></p> <p> <b>Never use the footrest at any other angle than 0 degrees or 90 degrees. Using the footrest at any angle between 0 and 90 degrees may cause an injury or an accident as the footrest may suddenly fold.</b></p>	
<p>1. Use the footrest only at 90 degrees when the chair is in the upright position and only at 0 degrees when in the supine position.</p>	
<p><b>Adjusting the Leg Rest</b></p> <p> <b>Always use both hands when raising or lowering the leg rest. Do not snap the lever - snapping may lead to mechanical problems.</b></p>	
<p>1. Hold the footrest guard handle with one hand.</p>	
<p>2. Raise the leg rest slowly by holding and pulling the leg rest adjusting lever and guard handle with the other hand.</p>	
<p>3. To lower the leg rest, hold the footrest guard handle with one hand. Lower the leg rest slowly by holding and pulling the leg rest angle adjusting lever and guard handle together with the other hand.</p>	

## Adjusting the Convertible Chair Casters

<p><b>The Convertible Chair has three caster functions:</b></p> <ol style="list-style-type: none"> <li>1. Lock Function</li> <li>2. Free Function</li> <li>3. Steering Lock Function</li> </ol> <p> <b>The caster lock is not designed to stop the Convertible Chair in motion. Do not operate the caster control panel while moving the equipment.</b></p>	
<p><b>Caster Lock Function</b></p>	
<p>1. Use this function to hold the Convertible Chair motionless when transforming it from a supine to an upright position and when transferring a patient to and from a bed. Step down towards you to initiate.</p>	
<p><b>Caster Free Function</b></p>	
<p>1. Use this function to bring the Convertible Chair alongside a bed or to "freewheel". This function allows all four wheels to swivel and rotate freely. Engage neutral position to initiate.</p>	
<p><b>Steering Lock Function</b></p>	
<p>1. Use this function to lock the two front casters from swiveling so that the Convertible Chair can easily be steered in a straight line. Push pedal forward to engage steer function.</p>	

## Adjusting the Convertible Chair Brackets

<p><b>The Convertible Chair brackets are installed on each side of the headrest and footrest.</b></p> <p> <b>Use the chair brackets only when in the supine position. Protruding chair brackets present a great danger as people and objects may get caught on them. After transferring a patient, be sure to restore the chair brackets to their original position before transforming the equipment to the upright position.</b></p> <p> <b>Use the chair brackets only when transferring the patient using the transfer bar otherwise mechanical failure may result.</b></p>	
<p><b>Adjusting the Chair Bracket</b></p>	
<p>1. Bring the Convertible Chair to a supine position alongside the bed and lock the casters.</p>	
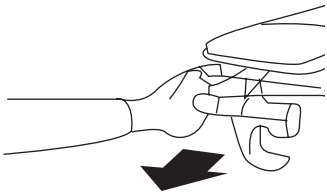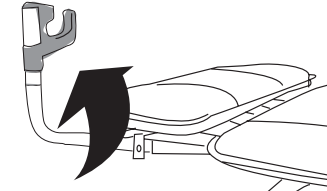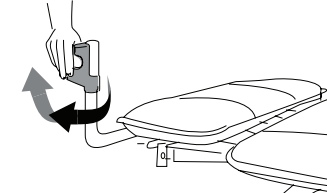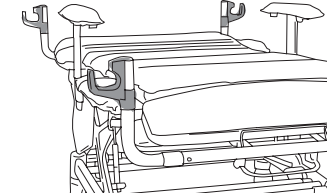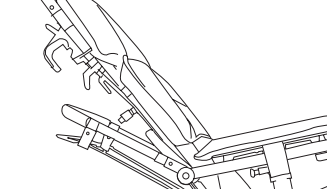
<p>2. Hold the chair bracket opposite the bed side with one hand and the plastic ring on the guard handle with the other.</p>	
<p>3. Turn the chair bracket 90 degrees while pulling the guard handle with the other hand.</p>	
<p>4. While holding the upright chair bracket with one hand and the plastic bearing with the other, pull and twist the chair bracket away from the bed and release it. Repeat for all brackets.</p>	
<p>5. When finished the chair brackets should be as shown and will be ready to install the PTS (Patient Transfer System).</p>	
<p>6. To restore the chair to its original position, reverse above steps. Caution: When restoring the chair brackets to their original position, always ensure that the plastic bearings are down.</p>	

## Operating the Convertible Chair

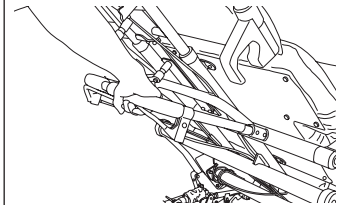
The Convertible Chair has unlimited positioning capabilities, including the Trendelenburg position.


Always use both hands when positioning the chair. Do not snap the levers - snapping may lead to mechanical problems.

### Tilting the Convertible Chair

1. To tilt the chair down, hold the headrest handle with one hand and squeeze the tilting lever and push handle together with the other hand. Push both slowly downwards or upwards.

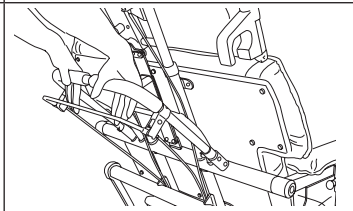
2. Release the tilting lever when the tilting reaches the desired angle.


### Sitting to Supine Position

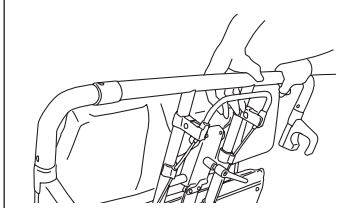1. To change from a sitting to the supine position, hold the headrest handle with one hand and squeeze the reclining lever on the headrest handle together with the other.

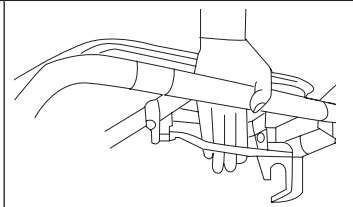
2. Push both slowly downward. Release the reclining lever when the back reaches the desired angle.


### Supine to Sitting Position

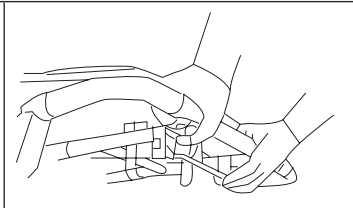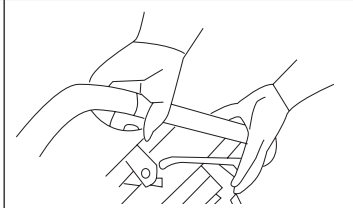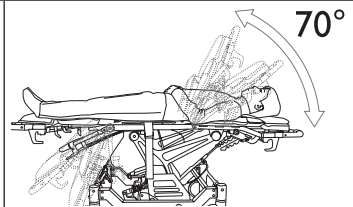
1. Hold the headrest handle with one hand and squeeze the reclining lever and headrest handle together with the other.

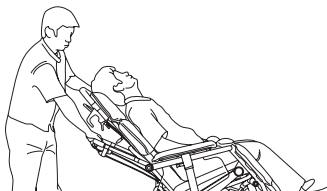
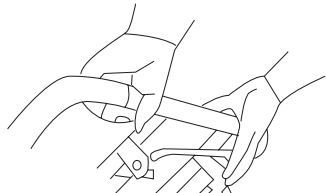
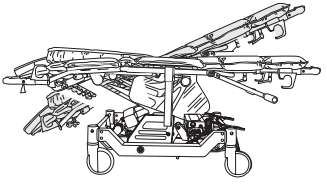
2. Lift slowly upward. Release the reclining lever when the back reaches the desired angle.

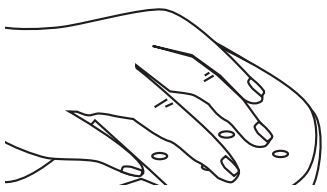
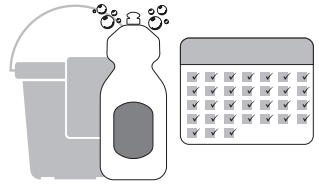
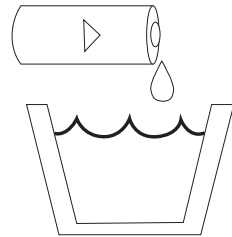
Note: Back adjustment angles can range between the supine position and 70 degrees.

<b>Trendelenburg Position</b>  <b>It is recommended to secure patient in chair before positioning in Trendelenburg position. Ensure that the tilt position is in its most forward position before putting patient in Trendelenburg to prevent patient from sliding towards the ground and out of the chair.</b>	
1. Hold the headrest handle with one hand and squeeze the tilting lever and push handle together with the other hand. Push both slowly downwards to full back tilt-in-space recline position.	
2. Hold the headrest handle with one hand and squeeze the reclining lever and headrest handle together with the other.	
3. Pull down to Trendelenburg position.	
 <b>Lower patient slowly and deliberately to prevent patient from sliding back towards the caregiver and to the ground when going into Trendelenburg position.</b>	

**Cleaning Recommendations**

 <b>The materials used to manufacture SOFlux LVT® and the performance of the fabric may be adversely affected by permitting any cleaning agent to remain in contact with the fabric for a prolonged duration or a failure to adequately rinse the agent from the fabric. Before using any cleaning product, test it on an inconspicuous area of fabric.</b>	
<b>Routine Care</b>	
1. Do not use harsh cleaners or solvents. Wipe fabric clean with a neutral soap and lukewarm water. Machine laundering or drying is not recommended. Do not pressure wash.	
2. After washing, wipe over with Sodium Dischloroisocyanurate (NaDCC) at 1,000ppm of available chlorine (or other chlorine releasing agent). Alternatively wipe over with a 70% Isopropyl Alcohol wipe. Dry thoroughly.	

3. For hard to clean areas, soft sponge with a liquid cleaner as specified on the manufacturer's product label.	
4. To control or prevent odors on long term incontinent applications, clean daily. The use of a scented cleaner / disinfectant is recommended.	
5. Do not use Hypocarbonate or Phenol based cleaning agents, any volatile liquids such as thinners or petroleum, or any abrasive compounds or pads when cleaning the chair.	
<b>Disinfection</b>  <b>Iodophor type disinfectants (for example, Betadine) will stain fabric.</b>	
1. Most phenolic or quaternary type disinfectants can be used. Dilute in accordance with manufacturer's instructions. Note: Undiluted Dettol may damage paint and plastic parts if prolonged contact is permitted.	
2. Gloves and protective clothing should always be worn when carrying out cleaning procedures in accordance with facility guidelines.	
 <b>Important!</b> When equipment has been used by a patient known or suspected of having an infection, this equipment must be disinfected before subsequent usage.	
<b>Blood Contamination</b>	
1. SOFluxLVT® can be disinfected with a 1:10 dilution of household bleach (5.25% sodium hypochlorite) as recommended by the CDC <sup>1</sup> . Weaker dilutions (example 1:100) may be used, but may not be in accordance with CDC recommendations.	

<sup>1</sup> US Department of Health and Human Services (February 1989)

## Material

Fabric information is offered as being typical for the indicated product by the manufacturer of the fabric. The values are based on laboratory results and are offered for guidance only. Lot to lot variations may occur that are not material to the performance of the product.

### Outside Fabric

SOFlux LVT (low vapor transmission) is a 6.75oz/sq. yard 70 Denier/210 thread count nylon fabric with a blended neoprene/butyl rubber coating. The product is designed for applications where a severe reduction of the Moisture Vapor Transmission Rate (MVT) is desired for the benefit of infection control.

### Outside Fabric Technical Data

EPA Registration Bacteriostat	10466-28
EPA Registration Fungistat	10466-23
Antibacterial	AATCC Method 147-1988 Saphylococcus aureus Contact inhibition 99%+
Antifungal	AATCC Method 30-1988 Saphylococcus aureus Contact inhibition 99%+
Primary Skin Irritation	Draize Dermal Toxicity, Non-Allergenic
Flame Resistance	NFPA 701 LS - Pass CAL 117 - Pass
Cigarette Ignition	FF4-72/16CFR, Part 1632 Pass (as a component in a finished mattress)
Moisture Vapor Transmission	ASTM E96/Method A .5g/m <sup>2</sup> /hr
Hydrostatic Burst (PSI)	Federal Standard 191-5512, 135lbs/61.2kg
Tear Strength	Federal Standard 191-5134 Warp, 4.6 lbs/2.1kg
Break Strength	Federal Standard 191-5100 Warp, 130 lbs/in (59kg/2.54cm) Fill, 105 lbs/in (47.6kg/2.54cm)
Abrasion Resistance	ASTM D 3886-92, 1300 cycles

### Interior Fabric

Cushion material is a urethane foam, which is the same material used in most hospital mattresses. Not only does it prevent bacterial growth, but is also not susceptible to damage from short term wet conditions. This material is also ideal for preventing pressure ulcers from extended immobility.

### Frame

The frame is composed of steel tubing and aluminum and plastic parts. Ensure all moisture is wiped off parts to avoid rusting and noise.

## Troubleshooting

Check the following

Problem	Check	Correction
Does not move	Caster lock	Release the caster lock (p.10)
Does not go straight	Position of caster control panel	Use the steering lock function (p. 10)
Headrest will not lock	Lever position	Correct the lever position (p.8,9)
Angle of leg rest not adjusted	Lever position	Adjust leg rest (p.10)

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