



8060

Posey® Bed

Where Comfort Meets Safety

User Manual



U.S. Pat. No. 7,047,991 & 7,383,849

H&R HEALTHCARE

Support Surfaces and Negative Pressure Wound Therapy
1750 Oak Street • Lakewood, NJ 08701 • (800) 801-5533 • www.handrhealthcare.com

How to Use this Manual

Introduction

This User's Manual is intended for those providing care to patients in the Posey Bed 8060 who are successfully trained on the use of the Posey Bed 8060. It is important that the person(s) using the Posey Bed be properly trained before using the bed.

In addition to this manual, you should also review:

- The Assembly Instructions
 - The In-Service Training Manual
-

Symbols

The symbols below are used throughout this manual and should be carefully read.

WARNING

A Warning alerts you to a safety hazard or unsafe practice, which, if not avoided, could result in serious injury or death.

CAUTION

A Caution alerts you to a hazard, which, if not avoided, may result in minor or moderate injury to you or the patient, or damage to the Posey Bed 8060.

NOTE

A Note provides helpful information to make bed use easier and more efficient.

Definitions

The following words have special meaning when used in this Manual:

BIOHAZARDOUS – matter that is contaminated with human blood; tissues; certain body fluids. Check with the patient's doctor if you have any questions about this hazard.

CARE PLAN – the patient's Individualized Care Plan (ICP) as written by the IDHT (see below).

INTERDISCIPLINARY HEALTHCARE TEAM (IDHT) – writes the Care Plan and supervises patient care. The team may include, among others, Doctors, Nurses, Occupational Therapy, Physical Therapy and Social Services.

ENTRAPMENT – an event in which a patient is caught, trapped, or entangled in a confined space. This can result in serious injury or death from suffocation, compressions, or lack of proper blood flow.

Need help?

If you need help or have any questions, please call Posey Customer Support at 1.800.447.6739 between 6:00 am PST, and 5:00 pm PST. You can also contact your authorized bed dealer.

Table of Contents

Topic	See Page
Welcome to the Posey Bed 8060	4
About the Posey Bed 8060	5
When the Bed Should Not Be Used (Contraindications)	6
A Special Note	7
Patient Reassessment	8
Description of the Posey Bed 8060	9
Parts of the Posey Bed 8060 System	9
Choosing the Right Bed for use with the Posey Bed 8060	10-11
Warnings and Precautions	12-14
Basic Bed Operation	15-18
Install or Replace Mattress	15
Zip and Unzip the Access Panels	16-17
Use the Tube and Drainage Ports	18
Before First Time Use	19-20
Check for Proper Bed Assembly	19
Guidelines for Proper Bed Location	20
Guidelines for Training	20
New Patient Selection and Monitoring	21
Emergency Patient Access and Exit	22
Regular Daily Patient Care	23-27
Moving the Patient Into the Bed	24
Access and Care for the Patient in the Bed	25
Moving the Patient Out of the Bed	26
A Special Considerations for Home Care Enviroments	27-28
Quick-Check 10	29
When Not to Use the Posey Bed	30-31
Quick Clean the Posey Bed 8060	32
Separate Mattress Compartment from Patient Compartment	32
Machine Washing the Posey Bed 8060	33
Posey Accessories	34
Troubleshooting and Frequently Asked Questions	35
Posey Limited Warranty	36
Posey Technical Service	36

Welcome to the Posey Bed 8060

Introduction

The Posey Bed 8060 is an A-Frame canopy system. When attached to a compatible hospital bed (not included), the 8060 is designed to help provide a safe, controlled environment for patients at extreme risk of injury from a fall or unassisted bed exit.

Prescription Only and Required Training

Because the Posey Bed 8060 is a restraint, its use must be prescribed by a licensed doctor. Anyone caring for a patient in the Posey Bed 8060 must be trained.

Benefits

The Posey Bed 8060 was designed to help provide these benefits:

- A controlled patient environment,
 - Reduce the risk of unassisted bed exit,
 - Reduce the risk of patient falls,
 - Avoid the risk of a patient being trapped under a mattress, and
 - Eliminate the need for side rails when the patient is alone. This avoids the risk of the patient being trapped in the side rails, or between the mattress and side rails.
-

Risks

Improper use of the Posey Bed 8060 may lead to serious injury or death. Patient monitoring should be determined by hospital protocol, a doctor, and the patient's care plan. As with any less restrictive restraint system, it is important to understand when the Posey Bed 8060 is needed, when it should not be used, and the dangers related to entrapment, suffocation, choking, and falls.

About the Posey Bed 8060

Indications for Use (When the Posey Bed 8060 can be used)

The Posey Bed 8060 is an A-Frame canopy system. When attached to compatible hospital bed (not included) the 8060 is designed to help provide a safe, controlled environment for patients at extreme risk of injury from a fall or unassisted bed exit. The Posey Bed 8060 is a less restrictive alternative to physical restraints such as belts, vests, or jackets for patients at least 46 inches tall, weighing between 46 and 300 pounds. The Posey Bed 8060 is a restraint, and must be prescribed by a licensed physician that includes Rx use in the home environment.

Bed Use

The Posey Bed 8060 must follow:

- All state regulations,
 - Federal Center for Medicare & Medicaid Services (CMS) Regulations,
 - The patient's care plan (check with the doctor if you have any questions about whether a patient's care plan is needed to help meet any special needs (if applicable) of the patient,
 - The policies of your facility (if applicable).
-

Patients Who May Benefit

Patients who may benefit from the use of the Posey Bed 8060 include patients who are at extreme risk for a fall and/or unassisted bed exit including those with:

- Severe osteoporosis, (brittle, fragile bones),
 - Severe risk of broken bones due to clinical conditions,
 - Lack of muscle, fat, and subcutaneous tissue to absorb the impact of a fall,
 - Use of blood thinners,
 - A history of falls in the past,
 - Conditions that cause uncontrollable movement such as Huntington's Disease.
-

Clinical Conditions

Certain patients at-risk for a serious injury from a fall may also benefit from the use of the Posey Bed 8060. This includes patients with behavioral, physical, mental or other symptoms identified by the IDHT such as:

- Musculoskeletal problems that cause an unsteady gait or impaired strength or biomechanics,
 - Altered awareness due to medication or neural illnesses or conditions that impair alertness and balance, and/or dementing illnesses,¹
 - Visual or peripheral numbness or weakness that impair positional sense.
-

Falls

Most falls occur in Neurology and Medical units, with the highest rate in Oncology units. Fall rates may be higher in units with higher patient to nurse ratios.²

¹Dementia patients are at increased risk of falls. This may be due to impaired judgment, altered gait, and/or sensory deficits. Ellelt, Anne "Keeping Dementia Residents Safe" Assisted Living Consult, Sept/Oct 2007, page 20.

²Hitcho, Eileen B. et al., "Characteristics and Circumstances of Falls in a Hospital Setting" JGIM, Volume 19, July 2004.

When the Bed Should Not be Used (Contraindications)

When Not to Use the Device

It is important to note that Posey Bed 8060 is not suitable for every patient because every patient's diagnoses, or activity level are different, the Posey Bed 8060 is not suitable for every patient.

The Posey Bed 8060 is not intended for use with:

- Patients who weigh less than 46 pounds, or are shorter than 46 inches.
 - These patients are at risk of serious injury or death, and the Posey Bed 8060 has not been tested for use by such patients.
 - Patients who weigh more than 300 pounds.
 - This is the maximum load capacity of the Posey Bed 8060 canopy.
 - Patients diagnosed with any condition that may cause violent or self-destructive behaviors.
 - These could result in self-injury and/or damage to the Posey Bed 8060.
 - Violent moving of the body from side to side could cause the Posey Bed 8060 to tip over or move to a position where the patient is injured by contact with an object in the room.
 - Patients who rub excessively.
 - Patients with an excessive Pica eating disorder.
 - This refers to a person who may attempt to eat the canopy or padding of the Posey Bed 8060.
 - Patients who suffer from this disorder include adults, and it is the most common eating disorder among children who are developmentally disabled.
 - Patients who are, or who become, claustrophobic while in the Posey Bed 8060.
-

Possible Adverse Reactions

Although it is a less restrictive restraint, the use of the Posey Bed 8060 could cause emotional, psychological, or physical problems in certain patients. These include:

- A significant change in mental status, such as claustrophobia or depression,
- Sudden mood changes which lead to a change in medical conditions,
- An increase in aggression or self-destructive behaviors.

Get help from the doctor or the IDHT if these symptoms appear and do **not** improve as the patient becomes used to the Posey Bed 8060. You may need to find a different method of treatment.

A Special Note

Background

All hospital beds pose safety and entrapment risks. The U.S. Food and Drug Administration (FDA) received approximately 803 entrapment reports in the U.S. for standard hospital beds over a period of 24 years from January 1, 1985 to January 1, 2009.³

The FDA formed the Hospital Bed Safety Workgroup (HBSW) to develop solutions to reduce the risk of patient entrapment in hospital beds. The HBSW issued guidelines in 2006 describing seven potential zones of entrapment and recommendations to reduce life-threatening entrapments associated with hospital bed systems.⁴

The Posey Bed 8060 was designed to help eliminate each of the seven entrapment zones when the canopy is used with a compatible hospital bed at the lowest acceptable height (16 or 17 inches from the floor to the deck of the bed) when the head of the bed is down and the mattress is flat.

At-Risk Patients

- Patients who, because of mental or physical disability, cannot reposition or remove him or herself from a confined area in the bed.
- Any other patient identified by the doctor, the care plan, or Interdisciplinary Healthcare Team (IDHT) to be at risk of entrapment in the gaps created when the head of the Posey Bed 8060 is up.

Reducing the Risk of Entrapment

To help reduce the risk of entrapment, do the following:

- Keep the mattress flat.
 - If you need to elevate the head or body of an at-risk patient, while the patient is alone, keep the mattress flat and use a Posey Torso Cushion (Cat. 8025) to position the patient.
 - If you must leave the head of the bed up for an at-risk patient (for example, while watching TV or if called for by the doctor's order, or the patient's care plan, use the Posey Filler Cushions (Cat. 8021) and provide extra monitoring.
- Provide extra monitoring.
 - Monitor patient per facility's restraint protocol to ensure that the Posey Filler Cushions cannot be removed by an at-risk patient and that an at-risk patient cannot crawl under or around the Posey Filler Cushions.
- Avoid stretching the canopy material.
 - The canopy may stretch over time during normal use, or due to patients who try to escape. This could result in pocket areas on the inside of the canopy. Regular washing and drying should help reduce this condition. **DO NOT** use any canopy that has tears or damage and request a replacement.

Alternative Method of Treatment

If necessary, consult the doctor or the IDHT to find a different method of treatment for any at-risk patient.

³This report can be found at <http://www.fda.gov/ForConsumers/ConsumerUpdates/ucm164366.htm>

⁴Guidance for Industry and FDA Staff – Hospital Bed System Dimensional and Assessment Guidance to Reduce Entrapment. March 10, 2006). This report can be found at <http://www.fda.gov/MedicalDevices/DeviceRegulationandGuidance/GuidanceDocuments/ucm072662.htm>

Patient Reassessment

CAUTION

If the patient's physical or mental health changes, check with the doctor to make sure the Posey Bed 8060 is still a safe environment. Use the Posey Bed 8060 only if there is not a less restrictive alternative. **ALWAYS** use the least restrictive device available for the shortest time while you seek a less restrictive option.

Alternatives to Restraint Use

Review the patient's mental and physical status often to be sure that the benefits of using the Posey Bed 8060 as a restraint outweigh the risks.

Safe use requires that:

- You define and remove, if possible, the cause of the symptoms that require a restraint. This may include catering to individual needs and routines; increased rehabilitation; restorative nursing; changing the environment; or increased supervision.
- You tailor interventions to the individual needs of the patient after weighing factors such as the patient's condition; behaviors; history; and environment.

When possible, seek guidance from the patient's family. Depending on the patient's condition, alternatives to restraints may include one or more of these:

- A bedside companion,
- Fall alarms,
- Floor cushions,
- Lowering the volume on the TV,
- Adjusting window shades or dimming lights,
- Soft music, or
- Use of a Posey Activity Apron (Posey catalog 7400)

As with all restraints, the Posey Bed 8060 should **NEVER** be used for nursing convenience.

Description of the Posey Bed 8060

Bed Features and Benefits

- **6 INCH (15 CM) ENCLOSED MATTRESS COMPARTMENT**
Helps avoid the risk of a patient being trapped under the mattress. The mattress compartment can be expanded to ten inches (25 cm) to fit a ten inch (25 cm) air type mattress.
- **NYLON CANOPY SYSTEM**
An A-frame green canopy surrounds the bed. The nylon is biocompatible. The canopy has four (4) access panels for easy access to the patient, and multiple ports for intravenous lines, the Nurse Call Button, or drainage line. Zippers are secured with Quick-Release Buckles.

Parts of the Posey Bed 8060 System

- 1. ENCLOSED EXPANDABLE MATTRESS COMPARTMENT.** Designed to fit a 6 inch (15 cm) mattress or a ten inch (25 cm) air mattress.
- 2. NYLON CANOPY.** The nylon material is water-repellent.
- 3. A-FRAME.** A sturdy "A" shaped metal frame supports the Nylon Canopy.
- 4. TWO "U" SIDE PANELS.** One (1) panel on each side with a single zipper on each panel for easy patient access.
- 5. HEAD (5A) AND FOOT (5B) ACCESS PANELS.** Helps provide easy access to the head and foot of the bed with a single zipper on each panel.
- 6. QUICK-RELEASE BUCKLES.** Clipped to zipper pull-tabs for added patient security.
- 7. FOAM PADDING.** Six (6) pads fit around the canopy frame to help reduce the risk of injury from contact with the frame.
- 8. TUBE PORTS.** Eight (8) ports, two on each of the four sides of the canopy to run tubes into the patient compartment.
- 9. DRAINAGE PORTS.** Two (2) ports, one on each side of the "U" Side Panels.
- 10. IV POST MOUNT BRACKETS.** Four (4) brackets, two on each side of the "A-Frame", to firmly attach the frame to the IV socket on the hospital bed.
- 11. POSEY BED INSTRUCTION STORAGE POCKET.** Stores all product Manuals and the "Quick-Check 10" for easy reference.



Choosing the Right Bed for use with the Posey Bed 8060

Compatible Hospital Beds

The Posey Bed 8060 is an A-Frame canopy that attaches to a compatible hospital bed. Because the canopy does not include a bed, it is very important to choose the correct hospital bed for proper and safe use. Using the wrong bed may result in serious injury or death to the patient. There are a number of hospital beds in use today, from many manufacturers, which are fully compatible with the Posey Bed 8060. Hospital beds come with either removable head and footboards, or non-removable head and footboards.

Compatible hospital beds must meet the following requirements:

- Length of bed frame must be $\leq 96"$ and does not retract $< 76"$
- Width of bed frame must be $\leq 36"$
- The height of the bed platform (deck of the bed) from the floor must be able to adjust between 16 and 25 inches.




Depending on the features of the bed, operational limits must also be observed when a Posey Bed canopy is in use as described in the User Manual. For example, the head of bed angle can only be raised to $\leq 70^\circ$.

It is important to keep the bed platform in the lowest position to keep the canopy tight and avoid excess material in the patient area when the patient is alone.

⚠ WARNING

NEVER use any of the features listed below while using the Posey Bed 8060 canopy:

- **NEVER** lower the deck of the bed below sixteen inches (16") from the floor to prevent suspending the mattress above the bed deck. The bottom of the canopy must be fully supported by the bed deck to prevent canopy damage; or
- **NEVER** raise the hospital bed above twenty five inches (25") from the floor to the deck of the bed; or
- **NEVER** tilt the hospital bed surface head-to-foot or foot-to-head (Trendelenburg feature); or
- **NEVER** allow the hospital bed to transform into a "chair" shaped position, where the foot of the bed can be lowered to an angle greater than five degrees (5°).

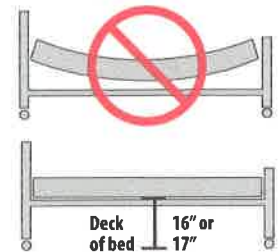

Using these features will damage the canopy system and bed motors and will increase the risk of serious injury or death from entrapment, suffocation from excess material in the patient compartment, or from an unassisted bed exit, which could result in a fall.


⚠ CAUTION

Fitting the Posey Bed 8060 through doorways

⚠ WARNING

The Posey Bed 8060 may not always fit through doorways. In this situation, lower the top piece to the lowest position, push the silver metal detent pin into the bottom hole, then return it back to the top hole after transport.

ALWAYS put the top rail back into the higher position, with the silver metal detent pin in the top hole, to keep the canopy taut and avoid excess material in the patient compartment, which could pose an entrapment risk.

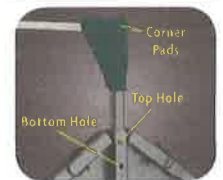

⚠ WARNING

Beds with removable head and footboards

ALWAYS use the IV Post Mount Brackets (comes standard with the Posey Bed 8060) on a hospital bed with removable head and footboards. Mounting the IV Post Mount Bracket to the IV Post Socket will firmly attach the A-Frame to the bed.

⚠ WARNING

Beds with non-removable head and footboards

ALWAYS use the Posey Bed Universal Straps (Cat. 8018, sold separately) for a hospital bed with non-removable head and footboards to firmly attach the frame to the bed. **NEVER** attach the Universal Straps to a bed with removable head and footboards. Doing so will cause the A-Frame to detach from the hospital bed when the bed is raised, and may result in serious injury or death from a fall or unassisted bed exit. **ALWAYS** use Posey Bed Headboard/Footboard Pads (Cat. 8002, Sold separately) to cover the head and footboards to help prevent injury from direct contact with solid head and footboards. Failure to do so may result in serious patient injury.

Choosing the Right Bed for use with the Posey Bed 8060 (Continued)

⚠️ WARNING
Incompatible Beds

NEVER use the Posey Bed 8060 with:

- Residential-type beds. The canopy system of the Posey Bed 8060 is not designed to securely attach to residential-type beds. A failure to heed this warning may result in serious injury or death from entrapment or suffocation, or from a fall or unassisted bed exit;
- Any hospital bed which cannot be lowered between sixteen (16") and seventeen (17") inches from the deck of the bed to the floor. It is necessary to lower the bed to at least seventeen to keep the canopy tight and avoid a "blousing" effect and excess material in the patient compartment, which will increase the risk of entrapment or suffocation;
- Any hospital bed that allows the head to be raised to an angle greater than seventy degrees (> 70°). Raising the head to a greater angle will damage the canopy and increase the risk of serious injury or death from a fall or unassisted bed exit; or
- Any hospital bed that does not securely attach to the Posey Bed 8060 with either (1) the four IV Post Mount Brackets or (2) the Posey Bed Universal Straps; as this may result in serious patient injury.

NEVER allow patient weight to exceed the maximum load of 300 lbs.

⚠️ CAUTION
Allen Head Screws

The Allen Head Screws on the bottom center piece of the Posey Bed 8060 must be secure, but loose enough so the A-Frame slides freely when raising or lowering the head or foot of the hospital bed. A failure to follow this warning may interfere with proper operation of the hospital bed.



⚠️ WARNING
Entrapment Risk (All Patients) - Use of Side Rails

NEVER leave side rails in the up position when the patient compartment is closed. A failure to follow this warning may result in serious injury or death from entrapment in the side rails, or between the mattress and the side rails. The risk of entrapment is even greater when the head of the bed is raised. These zones pose an extreme risk of serious injury or death from entrapment. An agitated or aggressive patient is also at risk of injury from contacting the side rails when they are in the UP position.



To help reduce the risk of entrapment, **ALWAYS** put side rails down when leaving a patient alone to reduce the risk of entrapment and serious patient injury or death. Side rails should **ONLY** be used as an aid to assist a patient in or out of bed, or to help prevent a patient fall while providing bedside care.

⚠️ WARNING
Entrapment Risk

ALWAYS lower the bed deck to 16"-17" (41 - 43 cm) from the floor and make sure the top rail is in the highest position before leaving the patient alone. This will keep the canopy tight and help prevent gaps or folds.

⚠️ WARNING

ALWAYS put the top rail back into the higher position, with the silver metal detent pin in the top hole, to keep the canopy taut and avoid excess material in the patient compartment, which could pose an entrapment risk.



Warnings and Precautions

⚠️ WARNING
Canopy Gaps



Entrapment risk for certain at-risk patients include canopy gaps. Raising the head of the Posey Bed 8060 will create “gaps” or “pockets” between the head of the bed and the canopy. These areas can cause extreme risk of serious injury or death from entrapment to certain at-risk patients. Keep the mattress flat with the head of the bed down, when an at-risk patient is alone. Use Posey Filler Cushions (Cat. 8021) if an at-risk patient’s head or body must be elevated (for example, while watching TV, or if called for by the doctor’s order or the patient’s care plan). In this case, provide extra monitoring per the facility’s restraint protocol to ensure that the Posey Filler Cushions (Cat. 8021) cannot be removed by an at-risk patient and that an at-risk patient cannot crawl under or around the Posey Filler Cushions. If filler cushions are not available, lower the bed to the flat position and use a Posey Torso Cushion (Cat. 8025).

⚠️ WARNING
Risks Associated with Suffocation or Choking

To help reduce the risk of suffocation, do the following:

- **NEVER** put a mattress inside the patient area,
- **ALWAYS** keep the mattress in the mattress compartment, completely zipped,
- Keep tubes or monitoring lines and the Nurse Call Button inside the bed only if allowed by the care plan.

⚠️ WARNING
Risk of Fall or Unassisted Bed Exit

NEVER use the Posey Bed 8060 if there is damage to the canopy, access panels or zippers. A failure to follow this warning may result in patient escape or unassisted bed exit, which may lead to serious injury or death from a fall. **ALWAYS** check the canopy and the zippers before leaving the patient alone to help reduce the risk of a fall or unassisted bed exit. (See page 14)

NEVER leave a patient alone if the zippers do not close completely, there are holes in the canopy or netting, the foam padding covering the metal frame is damaged or missing, or the frame is damaged. (See page 30)

⚠️ WARNING
Violent or Aggressive Patients

NEVER leave a patient in the Posey Bed 8060 if they have a risk of self-injury; injury to others; the bed tipping over; or if the patient tries to damage the bed.

- Be aware that a sudden mood swing may cause violent or aggressive behavior. Follow your facility’s security policy, the care plan or the doctor’s orders for such patients.
- A patient who may become violent or aggressive may require constant monitoring or different methods of restraint.
- No level of monitoring may avoid risk of serious injury to certain violent or aggressive patients, i.e. patients who flail from side to side attempting to tip over the bed or injure themselves against the frame.
- Use of the Posey Bed 8060 by an agitated patient or a patient who tries to escape must be approved by the doctor or the IDHT when tube(s) or line(s) are in use. There is a risk of serious injury or death if the patient becomes caught up, or disrupts the integrity of a tube or monitoring line.
- Watch for signs of claustrophobia, or a significant increase in stress or physical movement. Remove these patients from the Posey Bed 8060 **immediately** and notify the doctor. You may need to find a different method of treatment.



Warnings and Precautions (Continued)

⚠️ WARNING
Importance of Adhering to a Care Regimen

Failure to follow the doctor's order, the patient's care plan and all instructions, warnings and cautions in the manual and on all labels on the Posey Bed 8060, could result in patient death or serious injury. Monitor the patient's condition according to your facility's policy, the doctor's orders, and the care plans.

⚠️ WARNING

ALWAYS keep pets or small children a safe distance from the Posey Bed 8060 to help prevent injury to the patient, small child, or pet, if the bed is raised or lowered, and/or damage to the bed.

⚠️ WARNING
Importance of Patient Monitoring

NEVER leave a patient alone without proper monitoring. Failure to monitor per your facility's policy, the doctor's orders, or the care plan could result in serious injury or death.

⚠️ WARNING
Alterations to the Posey Bed 8060

NEVER change the Posey Bed 8060 for any reason.

- **NEVER** use any device, such as a Zip Tie or a padlock, to secure panel zippers. Only use the Posey Quick-Release Buckles. You must **ALWAYS** be able to access the patient quickly in an emergency.
- **DO NOT** add any accessories to the Posey Bed 8060 other than those recommended by Posey.

⚠️ CAUTION
Check the Canopy

Make sure there are no tears, holes or cuts in the nylon panels or netting and that the canopy and frame are secure and attached properly to the hospital bed.

NEVER leave objects in the bed that will allow the patient to damage the canopy, and possibly escape.

NEVER hang, suspend or attach toys or other items to the netting or canopy.

⚠️ CAUTION
Precautions During Transfers and Patient Care

Avoid risk of a fall or unexpected bed movement. Minimize the risk of injury to you and the patient by following the procedure (see page 24) for transferring patients in and out of the Posey Bed 8060.

⚠️ CAUTION
Access Panels

A fall may occur while one or more access panels are open during patient care. Make sure the patient is in a safe position on the mattress before you open an access panel.

⚠️ CAUTION
Daily Monitoring and Release Policy

Monitor the patient's condition per facility policy, the doctor's orders or the care plan. Explain the use of the Nurse Call Button to the patient and the patient's family.

Follow the care plan for daily patient release. Be sure to consider patient's needs for:

- Eating,
- Range of motion and rehabilitation,
- Walking,
- Recreation,
- Going to the bathroom, and
- Restorative nursing.


When used in the home, Posey recommends that the caregiver sleep in the same room. If you cannot be there in person, **ALWAYS** use something for monitoring the patient such as a child monitor, for constant audio or audio/video monitoring. Test to be sure that you can hear low levels of activity in the room.

Warnings and Precautions (Continued)

⚠️ WARNING Check the Zippers

- **NEVER** use the bed if a zipper slider is bent open or damaged and the zipper cannot be zipped completely closed. Remove the patient from the damaged bed and exchange it for a Posey Bed in good working condition. Send the damaged Posey Bed in for repair.
- **NEVER** use the bed if the zipper pull-tab is missing or broken.
- **NEVER** leave the patient's bedside until all access panel zippers are securely closed. Quick-Release Buckles must be connected and closed on both of the two (2) side access panels before leaving the patient alone. (See page 16)
- **NEVER** use the bed if a zipper coil is kinked, misaligned, or has gaps and does not close securely along the entire length.

Test that all zippers open easily and close securely along the entire length of the zipper. Inspect zipper coils for any kinks or misalignment. If any are identified, zip and unzip the zipper. If condition continues, **DO NOT** use product. Note the kinks on this sample.

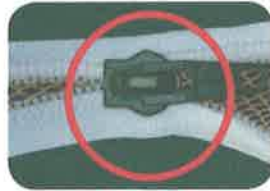


DO NOT use the Posey Bed if zippers have open gaps that do not close securely along the entire length



NEVER rip the panels open, as this will damage the zipper slider, preventing the zipper from closing securely.

Before leaving the patient alone, apply pressure with your hands along the entire length of the zipper to make sure the panel is securely closed and that there are no gaps or openings along the entire length of each zipper. Snap the quick-release buckles shut to ensure all zippers are securely closed, and check by tugging on each buckle.

ALWAYS use the zipper pull-tabs to open and close the zippers and ensure that the zippers are fully closed.

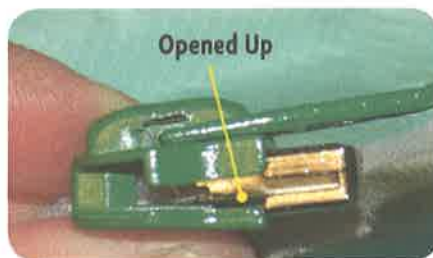

Damaged pull tab



Undamaged pull tab

Basic Bed Operation

Introduction

This section will address basic bed operations that you will do routinely when accessing and caring for a patient. These basic operations include how to:

- Install or replace the mattress,
- Zip and unzip the access panels,
- Use the tube and drainage ports.

Assembly Manual

Please refer to the Assembly Instructions (M5080) for detailed assembly instructions.

Bed Manufacturer's Instructions

Please refer to the Bed Manufacturer's Instructions for Use for proper operation of the bed. (Refer to pages 10-11 for Choosing the Right Bed.)

Install or Replace Mattress

WARNING

Choosing the right mattress

The Posey Bed 8060 mattress compartment is designed to fit either:

- A standard mattress that is six inches high, thirty-five inches wide and eighty inches long (6" x 35" x 80" (15 cm x 89 cm x 203 cm);
- An inflatable air-type mattress that is ten inches high, thirty-five inches wide and eighty inches long (10" x 35" x 80" (25 cm x 89 cm x 203 cm).

Any other size mattress may result in damage to the bed, patient injury or death.

How to install the mattress

1. Completely unzip the mattress compartment.
2. Insert the mattress.
3. Completely zip the mattress compartment closed.



Mattress Compartment Expansion

To fit the ten inch (25 cm) high air-type mattress in the mattress compartment:

1. Find the expansion zipper under the instruction storage pocket on the side panel.
2. Fully expand the mattress compartment by unzipping the zipper around the entire length of the mattress compartment.
3. Follow instructions above on how to install a mattress.



WARNING

To help reduce the risk of entrapment or suffocation:

- **ALWAYS** use the proper size mattress (see "Choosing the right mattress" above). Using a smaller mattress will result in gaps between the canopy and mattress. Using a larger mattress will stress the fabric and may damage the zipper.
- The mattress must be fully zippered into the mattress compartment at all times when a patient is in the Posey Bed 8060.
- **NEVER** open or close the mattress compartment while the head or knee of the bed is in the UP position, as this may damage the fabric and/or the zipper.
- **NEVER** use a mattress less than six inches (15 cm) high. **ALWAYS** make sure the expansion zipper is fully closed when using this size mattress.
- **NEVER** use a mattress smaller than ten inches (25 cm) high when the compartment is expanded.
- Make sure the expansion zipper is closed when using a six inch (15 cm) mattress.
- **NEVER** put a mattress inside the patient area of the bed.

A failure to follow these warnings may result in excess material in the patient area and increase the risk of serious injury or death from entrapment or suffocation.

Zip and Unzip the Access Panels

Introduction

The Posey Bed 8060 has four (4) zippered access panels each labeled:

- 2 side panels (A & B)
- 1 end panel (C)
- 1 end panel (D)

All panels unzip for easy patient access.

Using the Quick-Release Buckles

There is a Quick-Release Buckle at the start of the zippers on the side access panels. To open the buckle, pinch the 2 sides together and pull the clip apart.

To close the buckle, push the male and female ends of the Quick-Release Buckle together until you hear a click.

ALWAYS tug on the clip after closing to make sure it is connected.



WARNING

Make sure Quick-Release Buckles are securely fastened on both of the two (2) side access panels on the Posey Bed 8060 before leaving the patient's bedside. A failure to do so could result in serious injury or death from a fall or unassisted bed exit. **NEVER** use the bed if any of the Quick-Release Buckles are missing or damaged and do not close securely.

Access Panels

The Posey Bed 8060 has four (4) zippered access panels: two side panels, and two end panels.


Side Panels. Each side panel has two zippers and a pull-tab. Each zipper has a pull-tab for opening and closing zippers. At the top corner on each side of the bed are Quick-Release Buckles, a set of **WHITE** buckles and a set of **BLACK** buckles. The two **WHITE** buckles must fit together and the two **BLACK** buckles must fit together.

End Panels. Each end panel has two zippers and a pull-tab. For more security, place the zipper pulls in the overlap position, labeled "Hide Zipper Here" to keep tabs out of reach and sight of the patient.



Secure the Side Panels

To secure the side panel to the top of the frame, fold the mesh access panel over once and attach the Hook and Loop strips to one another. Fold the canopy over again and attach the back of the Hook and Loop to the sewn Hook and Loop strips on the top of the canopy. Repeat on other side and then roll the canopy at the middle and secure with the center support strap.

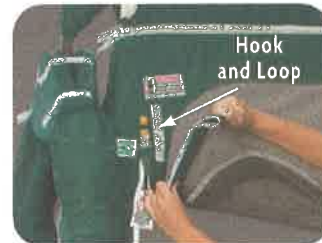

Zip and Unzip the Access Panels

How to Properly Zip and Unzip Panels

Each zipper has a pull-tab for opening and closing zippers. **ALWAYS** use the pull-tabs to open and close the zippers. **ALWAYS** make sure that all zipper teeth are engaged. Refer to page 14 for zipper warnings and precautions.



Zipper pull tab




Undamaged pull tab



Zipper kinks

Press against the closed zipped panel along the entire length of the zipper to ensure there are no gaps or openings before leaving the patient alone.

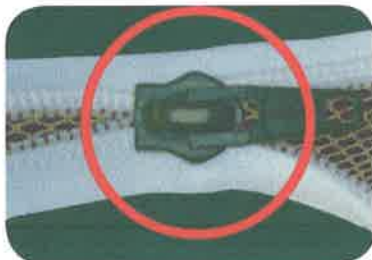

NEVER rip the panel open, as this will damage the zipper slider, preventing the zipper from closing completely.

DO NOT use a zipper with open gaps that does not close securely along entire length

Replacing Pull-Tabs

The side access panel zippers have a warning instruction plastic tag. If the plastic tag is damaged or becomes unreadable, it must be replaced. Call Posey at 1.800.447.6739 and request a free replacement (Part L9368).

WARNING

NEVER use the Posey Bed 8060 if there is damage to the canopy, damage to the access panels, or if the entire zipper does not close completely. A failure to follow this warning may lead to serious injury or death from a fall. **ALWAYS** check the canopy and ensure the entire zipper is completely closed before leaving the patient alone to help reduce the risk of a fall or unassisted bed exit.

Use the Tube and Drainage Ports

Introduction

Tube and drainage ports allow for tubes (food, drainage, etc.) and IV lines to be delivered to and from the patient while in the bed.

How to run Tube/ Line

To run a tube or line into the patient compartment, unzip the port, run the tube or line through the zipper opening into the patient compartment, and zip the port closed. Close the zipper as much as possible without impeding the flow of the tube.

A tube port in use:



A drainage port in use:



CAUTION

DO NOT place the tube through the netting.



WARNING

Make sure zippers are fully closed when ports are not in use. This will help reduce the risk of the patient damaging or opening a port, or trying to escape by putting a hand or foot through a port.

NEVER leave an unused port open when the patient is alone. **ALWAYS** check the condition of all tubes when raising or lowering the bed. There should be no “kinks” in the tube and fluids should flow freely to and from the patient.

Before First Time Use

Purpose

Before using the Posey Bed 8060 for the first time, it's important to check that the Posey Bed 8060 has been properly assembled, is in an appropriate area, and anyone who interacts with a patient in the Posey Bed 8060 successfully completes training.

3 Checklists

Use the following checklists before having a patient occupy the Posey Bed 8060:

- Check for Proper Bed Assembly
- Guidelines for Proper Bed Location
- Guidelines for Training

Check for Proper Bed Assembly

Purpose

To confirm that the Posey Bed 8060 was properly assembled and delivered in good condition.

Who Should Do This?

Biomedical Engineer or other authorized person.

Checklist

Prior to use with a new patient, photocopy and retain this checklist to record that each item has been verified.

NOTE: Each **No** (in table below) must be corrected before allowing a patient to use the Posey Bed 8060.

Check	Yes	No
1. Is the canopy metal frame unbent and unbroken?	<input type="checkbox"/>	<input type="checkbox"/>
2. Is the A-frame securely attached to the hospital bed using the four (4) IV Post Mount Brackets or the two (2) Universal Bed Straps?	<input type="checkbox"/>	<input type="checkbox"/>
3. Does the bed feel secure and free from any loose or moving parts?	<input type="checkbox"/>	<input type="checkbox"/>
4. Is the metal canopy frame completely covered by foam padding and no visible foam?	<input type="checkbox"/>	<input type="checkbox"/>
5. Is the canopy and netting clean and free from any tears, holes, or abrasions? (Page 12)	<input type="checkbox"/>	<input type="checkbox"/>
6. Are all zippers fully closed with zipper pins completely seated in their boxes and all teeth engaged? (Page 17)	<input type="checkbox"/>	<input type="checkbox"/>
7. Do the zippers close completely and open easily?	<input type="checkbox"/>	<input type="checkbox"/>
8. When pressure is applied along the entire length of the zipper, does the zipper hold securely without gaps or openings? (Page 17)	<input type="checkbox"/>	<input type="checkbox"/>

Guidelines for Proper Bed Location

Purpose

Patient safety asks that the Posey Bed 8060 is located in an area that provides easy access to the patient under both normal and emergency conditions.

Who Should Do This?

Attending Nurse or Caregiver.

Checklist

NOTE: Each **No** (in table below) must be corrected before allowing a patient to use the Posey Bed 8060. **Prior to use with a new patient, photocopy and retain this checklist to record that each item has been verified.**

Check	Yes	No
1. Is the Posey Bed 8060 clean?	<input type="checkbox"/>	<input type="checkbox"/>
2. Can a patient be easily accessed and can the bed be moved to the center of the room to allow access from all four (4) sides in an emergency situation?	<input type="checkbox"/>	<input type="checkbox"/>
3. Is the bed positioned so that the patient is NOT exposed to injury from contact with walls, furniture or other equipment?	<input type="checkbox"/>	<input type="checkbox"/>
4. Is the power cord out of the way to avoid a tripping hazard?	<input type="checkbox"/>	<input type="checkbox"/>
5. Is the bed away from all open flames, smoking materials or heat sources including baseboard heaters or space heaters?	<input type="checkbox"/>	<input type="checkbox"/>
6. Is the bed free of clutter or foreign objects?	<input type="checkbox"/>	<input type="checkbox"/>
7. Is the netting and canopy free from ANY sharp or hanging objects?	<input type="checkbox"/>	<input type="checkbox"/>
8. Is the netting and canopy free from ANY shipping or assembly damage to the bed or canopy system?	<input type="checkbox"/>	<input type="checkbox"/>
9. Are the hospital bed casters locked so the bed does not move?	<input type="checkbox"/>	<input type="checkbox"/>

Guidelines for Training

Purpose

Because the Posey Bed 8060 is a restraint system, anyone who cares for a patient in the bed must be properly trained.

New Patient Selection and Monitoring

Purpose

To make sure that the Posey Bed 8060 is right for the new patient and that his or her first hours in the bed are safe and secure.

Who Should Do This?

Attending Nurse or Caregiver.

WARNING

Before allowing a patient to occupy the Posey Bed 8060, confirm the following:

- A doctor has prescribed the Posey Bed 8060 for this patient,
- A current patient care plan is available and followed,
- The attending caregiver has been properly trained in the use of the Posey Bed 8060.

Checklist

NOTE: Any **YES** (in table below) indicates that the doctor must determine if the Posey Bed 8060 is appropriate for the patient. **Prior to use with a new patient, photocopy and retain this checklist to record that each item has been verified.**

Check	Yes	No
1. Does the patient weigh less than 46 pounds?	<input type="checkbox"/>	<input type="checkbox"/>
2. Does the patient weigh more than 300 pounds?	<input type="checkbox"/>	<input type="checkbox"/>
3. Is the patient less than 46 inches tall?	<input type="checkbox"/>	<input type="checkbox"/>
4. Has the patient been diagnosed with any condition that may cause violent or self-destructive behaviors?	<input type="checkbox"/>	<input type="checkbox"/>
5. Has the patient been diagnosed with an excessive Pica disorder or may attempt to eat the bed canopy or padding?	<input type="checkbox"/>	<input type="checkbox"/>
6. Has the patient been diagnosed with claustrophobic tendencies?	<input type="checkbox"/>	<input type="checkbox"/>
7. Has the patient been diagnosed as at-risk and cannot re-position himself or herself when the head of the bed is raised, therefore, requiring a flat bed or gap fillers?	<input type="checkbox"/>	<input type="checkbox"/>

Patient Monitoring

Monitor the patient's condition per facility policy, the doctors order or the care plan.

Regular Daily Patient Care

Go to: **REGULAR DAILY PATIENT CARE (PAGE 23).**


Emergency Patient Access and Exit

Purpose

Under certain conditions a patient may experience an emergency (such as respiratory failure or cardiac arrest). In this instance it is necessary to access the patient quickly and possibly remove the patient from the bed safely.

Procedure

NOTE: At least 2 people are needed for quick access and possible exit.

STEPS	DO THIS
1. Access the patient. 	a. Position the bed so it can be accessed from all four (4) sides. b. Lock the hospital bed casters. c. Lower the head and knee so the mattress is flat. d. Unclip the Quick-Release Buckles. e. Unzip ALL panels. f. Attend to the patient.
2. Exit patient, if necessary	Transfer patient to stretcher or wheelchair.



Position bed so it can be accessed from all four (4) sides



Lock the hospital bed casters



Unzip all panels

Disaster Planning

Should disaster strike such as an earthquake or fire, have a disaster plan ready.

Regular Daily Patient Care

Introduction

Patients using the Posey Bed 8060 should be carefully monitored and cared for according to the doctor's orders and the patient's care plan. Given normal, everyday care such as feeding or using the bathroom, this section addresses how to:

- Move the patient into the bed
- Access and care for patient in the bed
- Transfer the patient out of the bed
- Conduct the "Quick-Check 10" before leaving patient alone

Safe Transfers and Care

Safely moving the patient and completing patient care inside the Posey Bed 8060 includes a number of factors:

- Patient size
- Medical condition
- Disability
- Ability to move

Moving the patient to or from the bed will be different for patients who are able to stand instead of patients who must be lifted to or from the bed.

WARNING

NEVER use a patient lift for transfer unless it has been tested to make sure there is enough room under the canopy frame to do so safely. Using a patient lift that has not been tested may result in damage to the canopy. The clearance of the opening side access panel is 69" W x 50" H (175 cm x 127 cm).

NOTE: The frame at the bottom of the Posey Bed 8060 is open on one of the four sides.

ALWAYS access the patient from the open frame side. The open frame allows easy access for using lifts and/or scales.



Moving the Patient Into the Bed

Purpose

To safely get a new or existing patient into the Posey Bed 8060.

CAUTION


DO NOT use a patient lift unless already tested. This is to be sure there is enough room under the canopy frame to transfer the patient safely without damaging the canopy (see page 23).

CAUTION

During patient care and when moving the patient into and out of the Posey Bed 8060, lock all hospital bed casters.

Procedure

NOTE: Additional persons may be needed for a safe transfer.

STEP	DO THIS
1. Prepare bed for patient. NOTE: Raise bed for easy access. 	a. Make sure area around the bed is clear of any tripping hazards. b. Ensure the hospital bed casters are locked. c. Put mattress in the flat position. d. The caregiver must access the patient from the open frame side. e. Unclip the Quick-Release Buckle. f. Unzip the side panel and secure with the hook and loop tabs.
2. Transfer the patient to the bed.	a. Speak gently and assure them the bed is for their comfort and security. b. Move them to a safe area on the mattress.
3. Complete the patient's care plan as needed.	a. Attach any IV lines, drainage tubes, monitoring lines as required. (Page 18) b. Talk to patient to make sure they feel comfortable and secure at all times.
4. Close access panels.	a. Zip panels completely. Check that zipper is closed securely along the entire length of the zipper. (Page 14) b. Close the Quick-Release Buckle and tug on the clip to make sure it is connected. (Page 16)
5. Complete "Quick-Check 10" (see page 29).	

Access and Care for Patient in the Bed

Purpose

To access safely and provide care to a patient in the Posey Bed 8060 including feeding, IVs, catheters, respiratory therapy, positioning, visitors and medicating.

⚠ WARNING


Make sure the patient is a safe distance from the side panel and the mattress is flat before accessing them. Serious injury may occur due to the added distance if the patient falls when the bed is raised.

⚠ CAUTION

During patient care and when moving the patient into and out of the Posey Bed 8060, lock all hospital bed casters.

Procedure

NOTE: Approach the patient gently to ensure they are comfortable and secure.

STEP	DO THIS
1. Access the patient. NOTE: Raise bed for easy access. 	a. Make sure area around the bed is clear of any tripping hazards. b. Ensure the hospital bed casters are locked. c. The caregiver must access the patient from the open frame side. d. Unclip the Quick-Release Buckle. e. Unzip the side panel and secure with the hook and loop tabs.
2. Attend to the patient.	The caregiver should position his or her body to block the area of the open access panel to help reduce the risk of a fall while providing care.
3. Complete the patient's care plan as needed.	a. Attach any IV lines, drainage tubes, and monitoring lines as required. (Page 18) b. Talk to the patient to make sure they feel comfortable and secure at all times.
4. Close access panels.	a. Zip panels completely. Check that zipper is closed securely along the entire length of the zipper. (Page 14) b. Close the Quick-Release Buckle and tug on the clip to make sure it is connected. (Page 16)
5. Complete "Quick-Check 10" (see page 29).	

Moving the Patient Out of the Bed

Purpose

To safely assist a patient out of the Posey Bed 8060 so he/she may use the bathroom or other reasons.

CAUTION


DO NOT use a patient lift unless already tested. This is to be sure there is enough room under the canopy frame to transfer the patient safely without damaging the canopy.

CAUTION

During patient care and when moving the patient into and out of the Posey Bed 8060, lock all hospital bed casters.

Procedure

NOTE: Additional persons may be needed for a safe transfer.

STEP	DO THIS
<p>1. Access the patient.</p> 	<ol style="list-style-type: none"> Speak gently to ensure they are comfortable and secure. Check that they are a safe distance from the panel. Make sure area around the bed is clear of any tripping hazards. Put mattress in the flat position. The caregiver must access the patient from the open frame side. Unclip the Quick-Release Buckle. Unzip the side panel and secure with the hook and loop tabs. Detach tubes as needed.
<p>2. Assist patient to an upright position. NOTE: Be aware that a reclined patient may have equilibrium problems when arising – support as needed.</p>	<ol style="list-style-type: none"> Help patient slide both legs to the side of the bed. Change as needed depending on patient's needs. Consider gait belts / transfer belts to assist with patient transfer.
<p>3. Help patient out of the bed.</p>	<p>Continue to talk gently to the patient to reassure them.</p>

Special Consideration for Home Care Environments

Use in a Home Care Setting

All caregivers must be properly trained and familiar with the content of the manual before caring for the patient in a Posey Bed. Pay particular attention to all warnings and cautions in the manual to avoid injury or death.

- Only for the amount of time prescribed for use by a licensed medical doctor (Rx Only).
- Only if each caregiver has reviewed and understands all instructions and warnings for safe use.
- Only if persons are fully trained on the use of the Posey Bed 8060 and are present in the house to monitor and attend to the patient's needs whenever the patient is in the bed.

Dr.'s Orders

- You must carefully follow the doctor's orders and the patient's care plan (if any) for use of the Posey Bed 8060. This includes all orders for monitoring and patient release.

Care Plan


If your patient was using a Posey Bed during hospitalization or while in a nursing home, you should ask the facility for a copy of the Care Plan to help guide you in the safe use of the Posey Bed 8060. Check with the doctor to see if the Care Plan is necessary to help meet the special needs of your patient.

Entrapment Risk

Entrapment risks for certain at-risk patients include canopy gaps. Raising the head of the Posey Bed 8060 will create "gaps" or "pockets" between the head of the bed and the canopy. These areas can cause an extreme risk of serious injury or death from entrapment to certain at-risk patients. At-risk patients are:

- Patients who, because of mental or physical disability, cannot reposition or remove him or herself from a confined area in the bed.
- Any patient identified by the doctor, the Care Plan, or Interdisciplinary Healthcare Team (IDHT) to be at risk of entrapment in the gaps created when the head of the bed is up.

For at-risk patients, ALWAYS use Posey Filler Cushions (Catalog 8021, page 34) or Posey Torso Cushions (Catalog 8025, page 34). Refer to pages 7, 11 and 12 for detailed information about reducing entrapment risks, the use of Posey Filler Cushions and Posey Torso Cushions, and providing extra monitoring to ensure that the Posey Filler Cushions cannot be removed by an at-risk patient and that an at-risk patient cannot crawl under or around the Posey Filler Cushions.

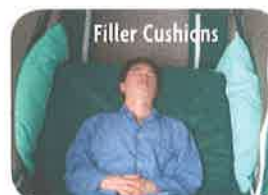

Special Consideration for Home Care Environments (Cont.)

⚠ WARNING

Entrapment Risk (All Patients) - Use of Side Rails

NEVER leave side rails in the up position when the patient compartment is closed. A failure to follow this warning may result in serious injury or death from entrapment in the side rails, or between the mattress and the side rails. The risk of entrapment is even greater when the head of the bed is raised. These zones pose an extreme risk of serious injury or death from entrapment. An agitated or aggressive patient is also at risk of injury from contacting the side rails when they are in the UP position.



To help reduce the risk of entrapment, **ALWAYS** put side rails down when leaving a patient alone to reduce the risk of entrapment and serious patient injury or death. Side rails should **ONLY** be used as an aid to assist a patient in or out of bed, or to help prevent a patient fall while providing bedside care.

In Patient Emergency

Immediately address the patient's needs or remove the patient from the bed. Contact the doctor or call 911.

⚠ WARNING

Importance of Patient Monitoring

If possible, Posey recommends a caregiver sleep in the same room as the patient. If you cannot be physically present to monitor the patient, you **MUST** use a device, such as a child monitor, for **CONSTANT** audio and/or video monitoring. Test to be sure that you can hear low levels of activity in the room. Failure to do so could result in serious injury or death.

“Quick-Check 10”

Introduction

Before leaving a patient alone, WALK AROUND THE BED to be sure that each condition is met.



- 1 **Exterior:** Area around the bed is clear of any tripping hazards.
Interior: Clean and free of clutter and hazards.
- 2 **Mattress (At-risk patients not able to reposition themselves):** To avoid entrapment, keep the head of the bed flat and use a cushion to elevate the torso if necessary. If the head of the bed is raised, use gap filler cushions.
- 3 **Compartments:** Zipped completely closed with no gaps or kinks in the entire length of the zipper. Test by applying pressure along the entire length of the zipper.
- 4 **Zippers and quick-release buckles:** Securely closed.
- 5 **Drainage/tube port:** If tubing and tube port is in use, all fluids are flowing freely; if tubing is not in use, port is zipped closed.
- 6 **Frame Condition:** Canopy, panel, and netting have no tears, holes or abrasions. Frame is firmly attached to the bed platform, undamaged, and fully padded. Top rail is in the highest position.
- 7 **Bed height:** Bed platform should not be raised above 25" (64 cm) or lowered below 16" (41 cm). Lower bed to 16" (41 cm) when patient is alone.
- 8 **Side rails: ALWAYS** ensure all side rails (if available) are in the fully down position.
- 9 **Casters:** all hospital bed casters (if available) are locked.
- 10 **A. Nurse Call Button:** If the patient's care plan allows it, the nurse call button is placed within reach of the patient.
B. Monitoring (Home Use): Caregiver should sleep in the same room or use a device, such as a child monitoring system for constant monitoring.

(See Reverse for Warnings.)

When Not to Use the Posey Bed

Purpose

To make sure that the Posey Bed is properly assembled and is in good condition before patient use.

⚠ WARNING

The Posey Bed **MUST** be inspected each time before leaving the patient's bedside. **ALWAYS** remove the patient from a damaged Posey Bed and remove it from service to prevent serious injury or death. Notify an authorized Posey representative for information about repairs.

⚠ CAUTION

The repairs must be completed before the bed can be put back in service.



- Canopy Metal Frame is bent or broken.



- Canopy is not securely attached using the IV Post Mount Brackets or the Universal Bed Straps.



- Canopy Frame is not completely covered by foam padding.




- Canopy and Netting have tears, holes, or abrasions.

When Not to Use the Posey Bed (Cont.)



- Zippers and zipper pins are not completely seated in their boxes and/or not all of the teeth are engaged.

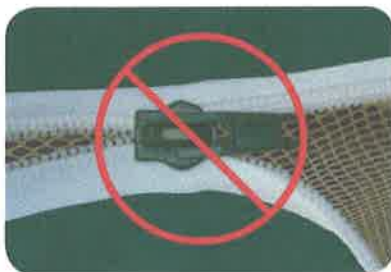

- Zippers produce gaps and/or openings when pressure is applied to the entire length of the zipper.



- Zippers do not close securely along the entire length. Gaps or openings in the zipper may allow the patient to open the zipper and fall out of bed. Always apply pressure along the entire length of the closed zipper to make sure there are no gaps or openings.



- Zipper pull tabs are open/damaged and will not close the zippers.



Quick Clean the Posey Bed 8060

Purpose

To keep the Posey Bed 8060 clean and disinfected between complete bed and canopy cleanings.

Procedure



STEPS	DO THIS
1. Wipe the fabric with cleaning agent NOTE: NEVER use a concentration of cleaning agent greater than recommended by the manufacturer.	<ol style="list-style-type: none"> Prepare a solution of diluted disinfectant or a water-based germicide. Use a damp cloth. Rinse with water to remove excess cleaning agent.
2. Dry fabric	Either wipe or air dry before use.
3. Clean the Hook and Loop strips	Use a stiff brush to remove any lint or foreign materials.

Separate Mattress Compartment from Patient Compartment

Purpose

To separate the mattress compartment from the patient compartment for easier loading in a washer.

Procedure

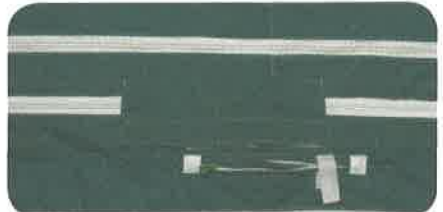STEPS	DO THIS
1. Open the flap below the side panel of the canopy to access the red zippers.	
2. Unzip the zippers completely to separate the mattress compartment from the patient compartment.	

Machine Washing the Posey Bed 8060

⚠ WARNING

ALWAYS wash the canopy between patients to help prevent transmission of infectious disease.

Purpose

To keep the Posey Bed 8060 clean and disinfected.

Procedure

STEPS	DO THIS
1. Wash the Canopy on one gentle cycle in cold water. NOTE: Water should be between 65-85°F (18-30°C) for approximately 25 minutes.	a. Use a mild detergent. b. Load panels loosely into washing machine. c. Rinse thoroughly with water.
2. Dry fabric.	Hang dry at room temperature or machine dry on air-only cycle (no heat).
3. Clean the Hook and Loop strips.	Use a stiff brush to remove any lint or foreign materials.
<ul style="list-style-type: none"> - DO NOT use industrial extractor-type laundry equipment, as this may damage the zippers and buckles. - NEVER use bleach, harsh cleaners or solvents. - ALWAYS attach hook and loop pieces together. Zip all zippers and connect Quick-Release Buckles. 	
4. Clean the bed frame. NOTE: This may be done with a solution of diluted disinfectant or water-based germicide.	Wipe frame undercarriage with a damp cloth. NOTE: Ensure cloth is damp, not wet.
<ul style="list-style-type: none"> - AVOID motor or bed controls when using water to clean bed frame, as this may damage the electrical components. - UNPLUG the power cord before cleaning bed frame. 	

⚠ CAUTION

⚠ CAUTION

Posey Accessories

The following accessories are appropriate for use with the Posey Bed 8060.

⚠ WARNING

NEVER use any accessory with the Posey Bed 8060 other than those recommended by Posey. A failure to follow this warning may result in serious injury or patient death.

POSEY HEADBOARD/FOOTBOARD PADS (8002)

To help protect the patient from injury from contact with the head or footboards of the hospital bed.



POSEY INCONTINENCE PAD (8005A)

To help protect mattress and bedding from urine spills. 8005A can be used with the Posey Support Surface.



POSEY SUPPORT SURFACE (8007)

To provide additional padding and to help secure sheets by tucking them underneath the support surface.



POSEY BED TRAVEL COVER (8008)

To provide a cover for the Posey Bed during transport and storage.



POSEY UNIVERSAL STRAPS (8018)

To firmly attach the bed enclosure frame to the bed (shown in black).



POSEY FILLER CUSHIONS (8021)

Recommended to help reduce the risk of entrapment or injury from gaps created when the head of the bed is in the UP position.



POSEY TORSO CUSHION (8025)

Recommended to elevate the torso when the head of the bed is DOWN.



POSEY BED PEDIATRIC CANOPY COVER (8003P) AND SKIRT (8003PS)

Attaches to the outside of the Posey Bed canopy to provide a child-friendly appearance.



Troubleshooting and Frequently Asked Questions

Canopy

1. **Q: Why is the zipper difficult to open or close?**

A: It could be one or more of the following reasons:

- Fabric, bedding, or other material caught in the zipper slider. To release material, back the zipper slider off by pulling the tab in the opposite direction until the slider is free. Make sure the canopy is intact and there are no tears or holes in the fabric.
- Check the zipper teeth to make sure none of the coils are kinked.
- Use the pull-tab to slide the zipper. Each access panel zipper has a warning instruction plastic tag. If the plastic tag is damaged or becomes unreadable, it must be replaced. Call Posey at 1.800.447.6739 and request a free replacement (Part L9368).

2. **Q: Why is the zipper separating after zipping up?**

A: The slider is damaged and does not properly engage the coils when it is being closed.

DO NOT use the Posey Bed 8060. This problem can be minimized by **ALWAYS** using the zipper pull-tab to open the zipper. **NEVER** rip the access panel open because it will damage the slider and prevent the zipper from closing securely.

3. **Q: Where do I find the serial number of the Posey Bed 8060?**

A: The serial number label is on the bottom right of one end of the bed and on the Metal Retainer.



Patient Lifter

Q: Can I use a patient lift with the Posey Bed 8060?

A: Yes, providing it has been tested to ensure there is enough clearance under the canopy frame. The clearance of the opening side access panel is 69" W x 50" H (175 cm x 127 cm) at bottom.

Disease and Self-Care Information

To help reduce the chance of transmitting infectious disease, all caregivers must wash their hands after patient care.

Register the Posey Bed

To register your Posey Bed, please visit our website at www.posey.com or call Posey Technical Services at 1.800.447.6739 and press "5." Be sure to let us know if you change your address.
