Garantía Limitada

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LUMEX®

Features:

- Anodized aluminum frame features a wider and increased depth frame that supports a 600 lb weight capacity
- Dual release folding mechanism allows frame to be folded easily and is ideal for users with limited dexterity
- Contoured, textured handgrips provide additional comfort
- Stable and lightweight frame makes it easy to maneuver

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For the most updated and current information on packaging, warranties, products and specifications, please visit our website at www.grahamfield.com

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© 2014, GF Health Products, Inc.
2935 Northeast Parkway, Atlanta, Georgia 30360
Made in China www.grahamfield.com

GF0900097RevC14

LUMEX®

Imperial Collection Dual Release Folding Walker

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Items 604070A, 604070W

User Instructions and Warranty Information

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Importante: Lea Antes de Uso

Instructions for Use – Please Read All Instructions Before Use.

Opening the Walker

1. Remove closed walker from carton.
2. Grasp one handgrip and pull away from center of walker to unfold frame.
3. Repeat with other handgrip.
4. Push handgrips and sides out away from each other until the release mechanisms lock into place.

Closing the Walker

1. Push downward on yellow release button and hold.
2. Grab one handgrip and push inwards towards center of walker.
3. Repeat for other handgrip.

Adjusting the Height

NOTE: The walker is packaged in its lowest position. Height adjustment should be made to desired height before operation.

Consult with healthcare professional for proper adjustment.

1. To adjust the height of the leg attachment, depress the adjustment button and slide upwards or downwards to the desired height.
2. Adjust the height of the walker by aligning all four leg attachments to the same position height.

NOTE: For walker with wheels, ensure the leg marked with a "L" is installed on the left side of the frame and the "R" is installed on the right side.

Maintenance

- For dust and dirt, wipe walker gently with damp, soft cloth.
- Inspect wheels (if applicable) periodically for wear or damage. Use a mild soap and water solution to clean, rinse with clear water, then dry with a soft cloth.
- Never use organic solvents, abrasive cleaners, or scouring pads on any part of the walker.
- Inspect Walker weekly for proper function and wear. If any component is not in proper working order or appears worn, contact your GF Health Products, Inc. distributor for repair and/or appropriate replacement parts.
- If handgrips are loose, DO NOT use. Contact your GF Health Products, Inc. authorized distributor.
- Replace any broken, damaged or worn items immediately.
- To guarantee the best service from your product, always specify Lumex replacement parts, available from your GF Health Products, Inc. authorized distributor.

For more information regarding GF Health Products, Inc. products, parts and services, please contact your distributor or visit us online at www.grahamfield.com.

Safety Guidelines – Please Read Before Use

1. Each individual should ALWAYS consult with their physician or other health care professional to determine proper adjustment and usage.
2. Before use, be sure that the walker is in its open and locked position.
3. All legs MUST be in contact with the floor at all times during use. This will ensure the walker is properly balanced.
4. This product has a maximum user weight limitation of 600 lbs. ALWAYS observe the user weight limitation on the hangtag.
5. Ensure handgrips DO NOT twist on walker – otherwise damage or injury may occur.
6. ALWAYS test to see that the walker and attachments are properly and securely locked in the open position BEFORE using.
7. After installation and before use, ensure that all attaching hardware is securely tightened.
8. ALWAYS use caution when using the walker on wet and slippery surfaces. DO NOT use the walker on stairs and steep inclines.
9. Before attempting to reach objects or pick them up from the floor by reaching down between your knees, place feet securely on the floor. Use EXTREME caution when reaching for any object.
10. GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
11. Inspect Walker weekly for proper function and wear. If any component is not in proper working order or appears worn, contact your Lumex dealer for repair and/or appropriate replacement parts.

WARNING

WARNING: DO NOT use this product without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

WARNING: If components are damaged or missing, contact your GF Health Products, Inc. distributor immediately. DO NOT substitute parts.

WARNING: Before use, ensure that all assembled parts are secure and all hardware is tightened.

WARNING: Care should be taken to ensure that all height adjustments are secure, and that wheels (if applicable) and moving objects are in good working order before using this walker.

WARNING: ALWAYS observe the user weight limitation on the hangtag of this product.

WARNING: Notice for California Customers
California Proposition 65 **WARNING:** This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.

WARNING: Be careful when reaching for objects when using the walker. Reaching for objects can change the weight distribution of the walker and may cause the walker to tip, resulting in injury or damage.

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Limited Warranty

The following warranty information is a SUMMARY of the complete Product Warranty provided by GF Health Products, Inc. For the complete warranty, please visit our website at www.grahamfield.com.

GF Health Products, Inc. (“GF”) warrants to the original retail purchaser or lessee (either is the “Customer”) only, that it will replace or repair components, at GF’s sole discretion, that are defective in material or workmanship under normal use and service. The warranty is for parts only and does not include labor or costs of shipping. All warranties are conditioned upon the proper use of the product strictly in accordance with applicable GF instructions. Product provided to users as rentals must be serviced by the provider and inspected prior to delivery. Each user must be properly trained on the operation and safety of this product prior to use. Within the guidelines set forth in this document, the frame is warranted for five (5) years and the normal wear components such as tips, wheels/casters, and handgrips are warranted for three (3) months. The applicable warranty period shall commence from date of shipment to the Customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of the warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Dealer from whom the Customer purchased the product within the applicable warranty period. If there is not a Dealer, you must contact GF directly by calling (678) 291-3207, sending a fax request to 770-368-2386 or by e-mailing a request to cs@grahamfield.com. Specific directions will be provided by the Customer Service Representative.

EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following: 1) Defects, damage, or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; 2) Products not installed, used, or properly cleaned and maintained; and 3) Labor or shipping charges. Other exclusions apply.

ENTIRE WARRANTY. EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF’S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED.

For the complete warranty for this product, please visit our website at www.grahamfield.com.

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- Si los mangos están sueltos, NO LOS use. Llame a su distribuidor autorizado de GF Health Products, Inc.
- Para el mejor servicio de su andador, siempre use partes de reemplazo de GF Health Products, Inc.