



1750 Oak Street, Lakewood, NJ 08701 (732) 367-5533 (732) 367-6231 fax
www.handrhealthcare.com

Perimeter Cover Instructions

1. Inspect product prior to use. Do not use soiled or damaged covers.
2. Position the mattress cover over the mattress making note of the ends labeled “Head” and “Foot.”
3. Thread each of the straps located along the edge of the cover through available openings in the perimeter of the movable section of the bed frame. This will allow the cover to remain properly fitted to the mattress as the entire bed, head section or foot section is raised or lowered. Avoid areas where the straps may interfere with the lowering and raising mechanism of the bed.
4. Secure the end of the quick-release buckles and tighten the straps securely.
5. If using a sheet, attach the eight sheet clips, located on the quick-release buckles, to the sheet.
6. If desired, attach the foot bolster to the mattress cover using the loop strip sewn into the cover.
7. Check for proper fit. Modified beds may not be compatible with Defined Perimeter Mattress Covers.

These instructions are for use with a standard size mattress, such as a Reduce Max or a Relief Max. Please note that a standard size Reduce Max is available with a built in perimeter cover.

When using a perimeter cover with any mattress larger than standard size, or any mattress that utilizes an air pump, such as a Signa, Relief Aire Alternating Pressure with Low Air Loss, Dyno or Relief Aire Low Air Loss, you must take off the mattress cover before positioning the perimeter cover – the cells will be exposed. Do not place the perimeter cover over the mattress cover. The perimeter cover will take the place of the mattress cover.

H&R Healthcare is dedicated to providing peace of mind through service, support and solutions. Please do not hesitate to contact your Regional Account Manager with any questions.